



Fitmate MED and all the accessories are provided together with a practical carrying case.



Fitmate MED uses either comfortable silicone masks for exercise and disposable Face Mask (for resting measurements).



Fitmate MED provides two flowmeters to ensure accuracy at any workload: one for exercise testing and one for resting measurements.



Easy to replace, the O₂ cell comes in a sealed bag. Lifespan is 12-18 months and indicated by the device.



Handle for Spirometry testing

© 2008/02 COSMED E & OE. Subject to alterations without prior notice. Part-No.: CO3226-02-93

Technical Specification

Main Functions

Tests	Exercise Testing (VO ₂ max), Spirometry, Resting Energy Expenditure, Body Composition, Risk Stratification, Standard measurements
Additional Testing - Software	Data management, Daily caloric intake, Trends

Oxygen Analyzer

Type	GFC (Galvanic Fuel Cell)
O ₂ Measurement range	0-22%
Gas Calibration	Automatic on room air
Warm-up time	None
Accuracy	±0.02%
O ₂ Sensor Lifespan	12-18 months
Sampling Type	Dynamic Mixing Chamber (international patent)
Sampling Rate	Programmable (minimum 15 sec.)

Flowmeter#1: Bidirectional digital turbine Ø 28mm (Exercise & Spirometry)

Ventilation range	5-300l/m
Flow resistance	<0.7cm H ₂ O/l/s@14l/s
Accuracy Flow/Volume	±2%

Flowmeter#2: Bidirectional digital turbine Ø 18mm (Resting)

Ventilation range	0-50l/m
Flow resistance	<0.7cm H ₂ O/l/s@3l/s
Accuracy Flow/Volume	±2%

Hardware

Dimensions & Weight	24 x 20 x 8cm / 1.5kg (9,4 x 7,9 x 3,1 in / 3.3 lb)
Display	Color LCD 320 x 240 pixel
Printer	High Speed Thermal printer 11 cm (4,3 in)

Standard Packaging

Fitmate unit, RMR - Flowmeter, RMR masks (10 pcs), Vo₂max - flowmeter, Silicone face mask, (M-size), Head cap for silicone mask, AC/DC Adapter, USB cable, Fitmate PC Software (CD-Rom), Skin-fold caliper, Body meter, Fitmate pedometer, Oxygen sensor, POLAR® HR belt and probe.

Available languages

Italian, English, German, Spanish, Greek, simplified Chinese, French, Dutch.

PC configuration required

Pentium or faster, Windows XP, VISTA 32, 128 Mb RAM or more, USB, CD-Rom reader, 80 Mb on HD space available.

Safety & Quality Standards

Equipment complies with MDD (93/42 EEC); EN 60601-1 (safety) / EN 60601-1-2 (EMC) FDA 510(k) cleared.




The first & unique diagnostic system for a complete Cardio Pulmonary Assessment

- ▶ Maximal oxygen uptake (VO₂max) and measured METs.
- ▶ Classification of Exercise Capacity & Anaerobic Threshold
- ▶ Nutritional Assessment and resting VO₂ for Fick equation
- ▶ Full Spirometry (FVC, SVC, MVV etc.)
- ▶ Multiple scores for Cardio Vascular and Pulmonary Risk analysis
- ▶ Body-composition & comprehensive Weight Management software
- ▶ Interfaces with conventional Stress testing ECG
- ▶ Affordable, compact & easy to use




Fitmate MED is the first diagnostic equipment designed to provide a full picture about Cardio Pulmonary Function. Fitmate MED measures maximal Oxygen Uptake that is the Gold standard for measuring Exercise Capacity and quantify Aerobic Fitness. Used together with exercise electrocardiography, Fitmate Med enables you to complete the cardiovascular stress test without the need for expensive investment, complicated procedures or specialist expertise. Rehabilitation of the patient is managed with exercise prescription and weight management software according to established international guidelines.

- ▶ Automatic control of the most common cycle and treadmill ergometers with cable interface

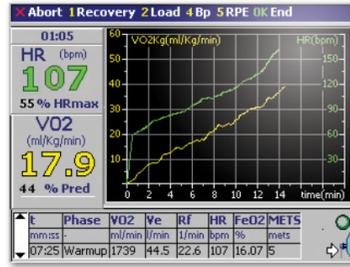

Maximal VO₂ Testing with both Treadmills and bikes

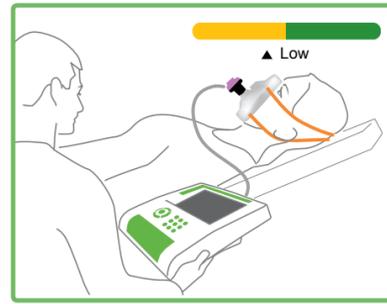

Resting Energy Expenditure Test



Spirometry Test


Resting Energy Expenditure

Fitmate measures oxygen consumption (VO₂) at rest with a, specialised low-flow flowmeter (supplied), ensuring high accuracy, comparable with conventional metabolic carts. Measurements include resting energy expenditure (REE), resting metabolic rate (RMR) and basal metabolic rate (BMR) in Kcal/day.



Spirometry

- ▶ Forced Vital Capacity (FVC)
- ▶ PRE & POST broncho-dilator response
- ▶ Maximum Voluntary Ventilation (MVV)
- ▶ Slow Vital Capacity SVC (Inspiratory & Expiratory)
- ▶ Automatic diagnosis complying with ERS and ATS recommendations

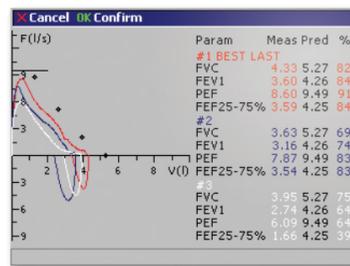

Clinical applications of VO₂

- ▶ Actual measurement of Exercise Capacity and METs (not-estimated)
- ▶ Pre-operative evaluation for surgical risk
- ▶ Classification and prognosis of CHF (congestive heart failure)
- ▶ Provision of objective selection criteria for heart transplant
- ▶ Decision tool for heart transplantations
- ▶ Differentiates Cardiac vs. Pulmonary limitations
- ▶ Determination of the intensity for exercise training as part of Cardiac Rehabilitation
- ▶ Measuring Resting Energy expenditure for providing optimal nutrition for recovery from illness and chronic health management
- ▶ Obesity treatment, and diabetes type II prevention
- ▶ Quantify severity and identify energy requirements for Respiratory disorders (COPD, Sleep disorders, Cystic Fibrosis)

VO₂max and real METS

- ▶ Choose between maximal and sub-maximal exercise protocols
- ▶ Real time display of pertinent respiratory parameters
- ▶ Automatic and manual detection of anaerobic threshold
- ▶ Pre-defined or custom exercise protocols (Bruce, cycle, ramp etc.)
- ▶ Automatic exercise training heart rate zones based on oxygen uptake

- Accurate** - Fitmate MED has been validated against the "gold standard" Douglas Bag technique for VO₂ measurements at rest and during exercise
- Portable, Desktop-size** - Compact, light, battery or mains operating.
- Built-in printer** - Integrated fast-thermal printer delivers high quality reports in a few seconds.
- Self-Calibrating** - Auto-calibration in less than 20 seconds using room air. No need for special gases or tanks.
- No warm-up time** - Plug it in, and Fitmate MED is ready for use
- No maintenance** - No on-site or self servicing required. Low-cost oxygen sensor replacement required after 12-18 months (automatic warning)
- Quality Control** - Warnings and quality control messages (mask leaks, breathing pattern etc.) are displayed during the test
- Software Included** - Windows XP/ VISTA compatible software for data management, trends and exercise prescription and weight management
- Face Masks** - 5 sized masks are available for exercise testing and one universal size disposable mask with antibacterial filter for REE testing
- Optional oxygen saturation** - SpO2 finger probe available for both rest and exercise testing


Proven Accuracy, Fitmate Validated vs. Douglas Bag

1) Validation of COSMED's Fitmate™ in measuring exercise metabolism. [David C. Nieman, at all. Research in Sports Medicine, 15: 1-9, 2007.]


2) Validation of Fitmate in measuring REE [David C. Nieman, at all. Research in Sports Medicine, 14: 1-8, 2006]

In conclusion, the Fitmate metabolic system accurately measures oxygen consumption during graded treadmill exercise as well as during Resting when compared with the Douglas bag system in male and female adults.

Both validations show an excellent correlation (R² > 0.98)

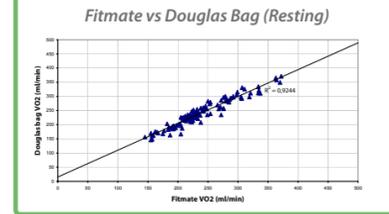

The graph shows VO₂, HR, markers and typed BP measurements

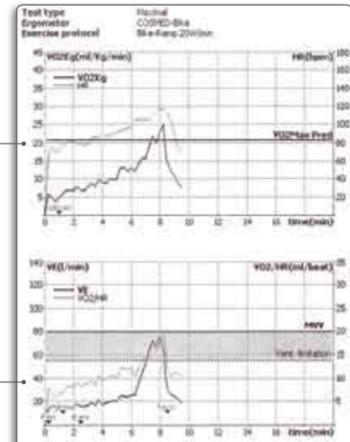


Ventilatory limitation area for easy interpretation

Gas exchange data compared to predicted

Gas exchange	Measured	Normal
Functional Capacity	7.2	8.0
Peak VO2 (ml/min)	3225	4055
Anaerobic Threshold (ml/min)	85	111
Cardiovascular Rating (ml/min)	48	107
Maximal VE (l/min)	157	194
Maximal VCO2 (l/min)	19.0	23.9
Estimated peak Cardiac Output (l/min)	13.7	17.1
Estimated peak Stroke Volume (ml)	117	144
Pulmonary VE1 (l/min)	0	0.42
VE1 (l/min)	0	0.42
Peak VE (l/min)	63.7	79.6
Respiratory Reserve (%)	53	65
Other Peak workload (watt)	0	0
Peak Speed (mph)	0	0
Peak Grade (%)	0	0

Exercise capacity and exercise intensity for Cardiac Rehabilitation Training

Exercise Intensity for Cardiac Rehabilitation (W)	Very Light	Light	Moderate	Heavy
HR (bpm)	50-67	66-110	111-139	139-163
Load (watt)	20-59	60-129	130-226	227-401
%VO2	30-40	40-55	55-70	70-85
VE (l/min)	10	15	20	25

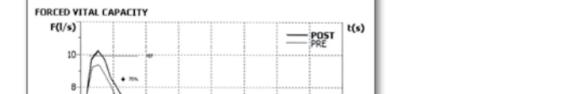


Parameter	UM	PRE	Pred	%Pred	POST	%PRE
BestFVC	l	4.45	5.25	85	4.84	109
BestFEV1	l	3.27	4.38	75	4.36	133
BestPEF	l/s	9.45	9.91	95	10.31	109
FVC	l	4.45	5.25	85	4.84	109
FEV1	l	3.27	4.38	75	4.36	133
PEF	l/s	4.40	5.55	79	10.31	109
MVV	l/s	9.45	9.91	95	10.31	109
MVF75%	l/s	7.23	8.49	85	7.55	104
MVF50%	l/s	3.63	5.54	65	5.29	461
MVF25%	l/s	0.72	2.58	28	2.88	401
FEF25-75%	l/s	2.30	4.90	47	4.93	214
FET100%	s	6.39	4.67	73	5.52	86
VEV1	ml	46	57	89	57	124
FEV1/FVC%	%	73	81	90	89	122
FEV1/FEV6%	%	74	83	89	89	122
LungAge	yrs	61	61	27	44	

Spirometry printouts showing data and graphs.

PC Software

A full PC software package is included, allowing the download and analysis of test data from Fitmate MED for advanced data management:

- ▶ Download data from Fitmate MED to PC
- ▶ Show trends, progress and deterioration
- ▶ Professional A4 or Letter format color printouts.
- ▶ Export Tests in different file format (ASCII, Excel etc.)
- ▶ Daily caloric intake (USDA Database)
- ▶ Compatible with any network on MS Windows environment

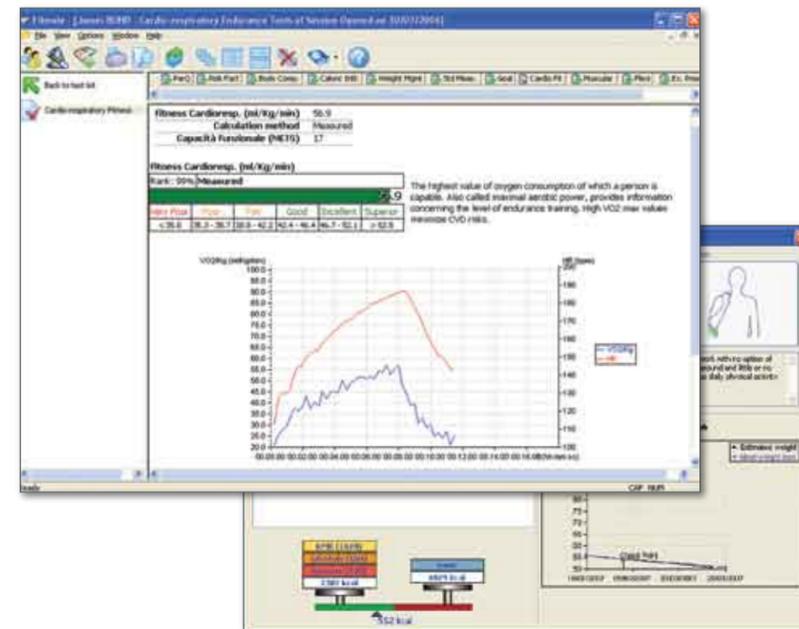

Personal Weight Management and Cardio-respiratory exercise Test

Risk Stratification

- ▶ Duke Treadmill Score
- ▶ Framingham Index
- ▶ European Heart Score
- ▶ Bode Index

Other Measurements

- ▶ Body composition measurement by skin-fold (caliper included)
- ▶ Individual weight management Programs based on Energy balance equation (built-in features)
- ▶ Standard Measurements (BMI, WHR BP, etc.)
- ▶ Daily caloric intake (software only)
- ▶ Lifestyle monitoring (pedometer and software tools included)