

Securin

Adhesive Remover Spray

Product Description & Intended Use

Securin Adhesive Remover Spray is a gentle, siloxane-based, alcohol-free spray. The intended use of Securin Adhesive Remover Spray is to support the removal of adhesive products from the skin.

Securin Adhesive Remover Spray enables painless removal of adhesive. The directional spray nozzle enables targeting the spray on the area of skin in focus thus reducing waste and aerosol exposure.

Securin Adhesive Remover Spray is intended to be used by trained medical professionals as well as by lay persons.

Indications

Securin Adhesive Remover Spray is indicated to support removal of adhesive products from the skin.

Contraindications

Securin Adhesive Remover Spray should not be used on persons with known allergy to the contents.

Securin Adhesive Remover Spray should not be used in open wounds.

Warnings

Securin Adhesive Remover Spray should only be used on infants under medical supervision. Securin Adhesive Remover Spray is intended for use on intact skin. If persistent skin irritation occurs in relation to the use of Securin Adhesive Remover Spray, please contact a healthcare professional. Avoid use on delicate, sensitive or non-intact skin or mucous membranes (e.g. wound beds, eyes, ears, nose, mouth, genitalia, anus, stoma), as it may cause skin irritation. Avoid direct inhalation. Keep out of reach of children. Keep away from heat, sparks, open flames, and hot surfaces. No smoking. Do not spray on an open flame or other ignition source. Protect from sunlight. Do not expose to temperatures exceeding 50C/122F.

Additional Information

In case of serious injury (accident) related to use of the product, please contact your local distributor or the manufacturer, as well as your local competent authority. To learn more or for technical assistance please contact us through www.furine.com. Furine ApS accepts no liability for any damage or loss that may occur if this product is used in violation to Furine ApS current recommendations.

Contents

Siloxane

Instructions for use

Spray Securin Adhesive Remover Spray around the edge of the adhesive product and wait a few seconds.

Gently peel the adhesive product from the skin using additional Securin Adhesive Remover Spray to ease removal. Wipe the skin with tissue afterwards, if necessary.

The product dries in few seconds and does not affect the adherence of subsequent adhesive products.

Disposal: The product should be disposed of in accordance with local regulations, e.g., together with normal household waste.



Legal manufacturer:
Furine ApS
Industrivaenget 1
3320 Skaevinge
Denmark



Storage conditions




Medical Device




Caution – consult
Instructions for Use




Non-sterile




Use-by date
See primary packaging



Batch code
See primary packaging



Keep away from sunlight




Recyclable
packaging




CE-marked device

Version: IFUA011-01UK
2025-01



Reference code
A011 & A012



Extremely flammable



Single patient
multiple use