

## Product Description & Intended Use

Securin Adhesive Remover Wipe (SAR Wipe) is a skin friendly, alcohol- and perfume free cleansing wipe. The intended use of SAR Wipe is to:

- prepare the skin before application of adhesive products by removing excess oil from the skin
- support removal of adhesive products from the skin after ended use

SAR Wipe is intended to be used by trained medical professionals as well as by lay persons.

## Indications

- SAR Wipe is indicated to prepare the skin before application of adhesive products.
- SAR Wipe is indicated to support removal of adhesive products and adhesive residues from the skin.

## Contraindications

- SAR Wipe should not be used on persons with known allergy to the contents.
- SAR Wipe should not be used in open wounds.

## Warnings

SAR Wipe should only be used on infants under medical supervision. SAR Wipe is intended for use on intact skin. Avoid use on delicate, sensitive or non-intact skin or mucous membranes (e.g. wound beds, eyes, ears, nose, mouth, genitalia, anus, stoma), as it may cause skin irritation. If persistent skin irritation occurs in relation to the use of SAR Wipe, please contact a healthcare professional. Keep away from heat, sparks, open flames, and hot surfaces. No smoking. Reprocessing, washing, disinfection, and/or (re)sterilization may impair the product's properties and pose an increased risk of physical injury or infection to the user. Furine ApS accepts no liability for any damage or loss that may occur if this product is used in violation to Furine ApS current recommendations.

## Additional information

In case of serious injury (accident) related to use of the product, please contact your local distributor or the manufacturer, as well as your local competent authority. To learn more or for technical assistance please contact us through [www.furine.com](http://www.furine.com).

**Contents:** Siloxane

## Instructions for Use

Preparation of the skin before application of adhesive products:

Tear the sachet to open and unfold the wipe.

Rub the wipe gently over the skin where the adhesive product is to be positioned.

Be sure that residue of prior applied skin products is completely removed.

Removal of adhesive products:

Begin loosening the edge of the adhesive product by gently rubbing with a SAR Wipe along the edge.

Continue rubbing with the SAR Wipe also under the adhesive product to release it from the skin.

If a wipe becomes dry during product removal, discard it, and use a new wipe.

Do not use wipes that have dried out.

After removal of an adhesive product the skin is cleansed of adhesive residue using a second SAR Wipe.

Disposal: The product should be disposed of in accordance with local regulations, e.g., together with normal household waste

