

Contents:

- 1 Furine[®] / Furine[®] plus / Furine[®] Small
- 1 Furine[®] Collect
- 2 Medical Skin Wipes
- 1 Securin[®] Skin Barrier

Product Description & intended use

Furine[®] is a non-invasive faecal management system that is applied to the perianal skin. The intended use of Furine is to divert faecal liquid matter from patients with faecal incontinence, thereby protecting the patient's skin and maintaining clean bedding. The liquid faecal matter seeps from the anal opening via Furine and into Furine[®] Collect. Furine may be used for up to 7 days before being replaced. The Medical Skin Wipe is used to prepare the skin before application of Furine to achieve stronger adherence. Securin[®] Skin Barrier is applied to form a protective barrier on the skin and enhances the adherence of Furine to the skin.

Indications

The device is intended for use on patients with liquid faecal incontinence to manage the collection of stools type 6 or 7 on the Bristol Stool Scale (BSS). Furine is intended to be used by trained medical professionals in a healthcare setting.

Contraindications

Furine may not be used on persons with:

- solid stools (BSS 1-5)
- large external haemorrhoids
- rectal prolapse
- perianal sores
- known allergy to the contents
- body weight less than 30 kg

Warnings

Do not re-use. Reusing Furine or parts of it can lead to infection or cross contamination. Discontinue use of Furine if there is perianal skin irritation or infection. Furine should only be used by persons with haemorrhoids under medical supervision. Furine should only be used on children under medical supervision. Do not sterilize Furine. Furine can lose its adhesive ability if the patient has a high degree of perspiration or is obese. Furine ApS accepts no liability for any damage or loss that may occur if this product is used in violation to Furine ApS current recommendations.












Contains

Medical Skin Wipes and Securin Skin Barrier contain siloxane.

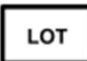
Additional information

MRI safety information: unknown. The presence of this device may produce an image artifact. In case of serious injury (accident) related to your use of the product, please contact your local distributor or manufacturer, as well as your local competent authority. To learn more or for technical assistance please contact us through www.furine.com.


Furine is a registered trademark

 Consult instructions for use	 5°C - 30°C Storage conditions	 NON STERILE Non-Sterile	 Single Use	 LOT Batch code See primary packaging	Version: IFUxB05-06UK 2026-03
 CE marked device	 LATEX Latex Free	 MD Medical Device	 REF Reference code See primary packaging	 Use-by date See primary packaging	 Legal Manufacturer Furine ApS Industrivaenget 1 3320 Skaevinge Denmark


Application Guide

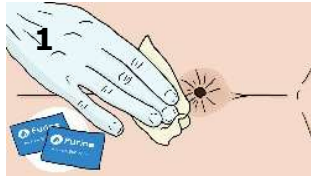

1 Cleanse and dry the perianal skin. Make sure that the skin is completely dry and intact. Prepare the skin by removing any excess oil with **Medical Skin Wipe** to ensure firm adhesion to the skin. **Do not** use any other skin barrier film products.

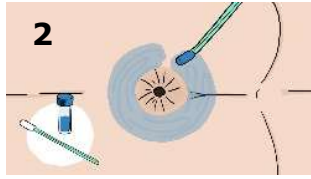


2 Apply Securin Skin Barrier to the skin around anus using the swab. It should be applied in a width of at least the size of the adhesive base which is to be positioned on the Securin Skin Barrier (approx. 3 cm). **NOTE:** The entire surface of Furine's adhesive base must be adhered to Securin Skin Barrier.

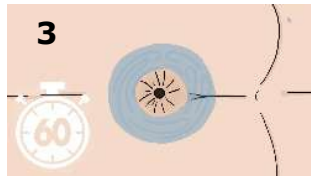

3 Allow the Securin Skin Barrier to dry for 60-90 seconds (**until dry**), while holding the buttocks separated. **Important:** It is important that Securin Skin Barrier is completely dry before Furine is applied. If it is wet or partially wet, there is increased risk of leakage.




4 Keep Furine securely fixed on the applicator by holding on to the strips of the adhesive base and remove the non-touch foil from the adhesive base.

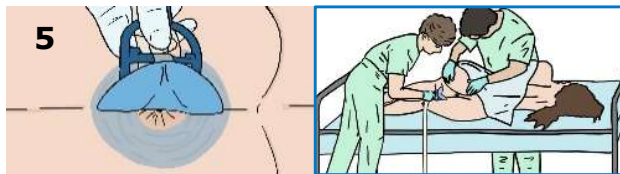

5 Use the applicator to position Furine on the perianal skin so the adhesive base adheres to the previously applied Securin Skin Barrier. Press Furine firmly to the perianal skin, so that it first has contact with crena ani and perineum.




6 Press the rest of the adhesive base firmly to the perianal skin using two fingers.

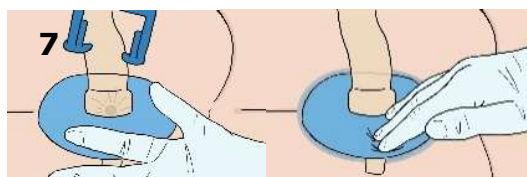

7 Carefully remove the applicator by pulling it away from the perianal skin, while at the same time, holding Furine in place with the other hand.

Press **again** the adhesive base firmly to the perianal skin using two fingers.

After Furine application, excess Securin Skin Barrier can be removed using the second Medical Skin Wipe provided.

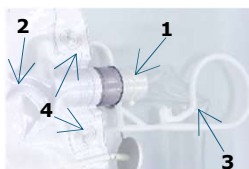

8 Connect Furine's connector (1) to Furine Collect (2) and insert the safety hooks (4) in the holes in Furine Collect. Position the bed-hanger (3) on the side of the bed.

Check the adhesive base for leakages regularly and repair with Securin Skin Barrier if necessary. Furine Collect must be replaced daily, or more frequently when needed.



9 After ended use, remove Furine using adhesive remover wipes or sprays recommended by Furine ApS such as Medical Skin Wipe, Securin Adhesive Remover Wipe or Spray. Carefully pull the strips of Furine while gently wiping with 2-3 adhesive remover wipes or spraying with adhesive remover spray. Be sure to carefully remove all residue of Securin Skin Barrier from the skin. **Do not** use wipes that have dried out. **Do not** use wipes or sprays that are not siloxane-based. Once used dispose of Furine in clinical waste as per local regulations.