



Product description

The baby EvacTAB is designed for the safe evacuation of premature and new born babies from hospital maternity wards and special care baby units in the event of a fire or similar emergency. The EvacTAB allows the baby to be carried to safety along with its medical Oxygen cylinder if required whilst having the rescuers arms and hands free; to tend to the child, open doors or negotiate stairwells safely.

Features

- Can carry up to 2 babies, depending on size and condition
- Enables carers to be “hands free” to administer urgent care to the baby and negotiate obstacles whilst evacuating to safety
- Has a secure storage pocket on the back of the tabard for a small oxygen cylinder to be carried along with the baby if oxygen is required for therapy
- Foam padded shoulders for comfort and stability
- Side fastening trident buckles for quick adjustment

- Heavy duty flame retardant canvas construction.
- Wipe clean PVC wall mounted storage bag

Specifications

Packed size:	60cm x 30cm x 3cm weight: 1.3kg
Deployed size:	Tabard: 80cm x 60cm, Pouch: 50cm x 30cm x 15cm
Safety tested to:	50kg
Working Load Limit (WLL):	30kg
Material Construction:	Heavy duty Flame Retardant Canvas, 33H foam padding, 50mm Hi Viz Yellow webbings with quick adjusting plastic buckles. Wipe clean PVC wall mounted storage bag.
Conforming Standard:	CE CA 015788 (MHRA)



EvacTAB



Using the baby EvacTAB

- Place EvacTAB tabard over head with carry pouch to the front and the trident buckles fastened and adjusted at the sides.
- If oxygen therapy is required, a second person can place an oxygen cylinder in the rear pocket, pass the supply hose through the shoulder loop and place in the front pouch.
- Place baby / babies in front pouch of EvacTAB. For extra protection wrap babies in their bedding before placing in pouch
- Move the baby / babies to safety in the EvacTAB, regularly monitoring them during the evacuation.

Testing & Cleaning

The EvacTAB should be inspected regularly for any signs of damage or excessive wear and replaced if necessary. If wiped or washed clean, any detergents or disinfectants should be well rinsed off and the EvacTAB dried before being replaced back in its wall mounted storage bag.

Further Information :



Contact – alison@3et.co.uk

Tel: (+44) 07585 541991

CE CA 015788 (MHRA)