



Product description

The EvacKRESH is designed for the rapid emergency evacuation of several infants from a crèche facility within a building. The EvacKRESH is quick and simple to deploy and load, it has 3 secure padded compartments with a light mesh cover, so the infants can be easily monitored during the evacuation. Up to 12 infants could be transported to safety depending on their size, the infant compartments are open space so there is no prescribed number of infants that can be placed in each compartment. Once placed in the EvacKRESH the infants are slid from the building by a minimum of 1 person in a horizontal evacuation and a minimum of 2 persons if the evacuation route involves moving up or down a flight of stairs.

Features

- Wipe clean wall mounted storage bag.
- Quick and simple to deploy in an emergency
- Quick and simple to load infants, no securing straps.
- No maximum infant capacity, infant number dependent on infant size and capacity of EvacKRESH compartment.
- Durable hard plastic base to move easily over nearly all types of floor surface.
- Easy clean Flame Retardant PVC construction
- Hi Visibility, soft touch pulling straps
- 50mm of 33H foam padding.

Specifications

Packed size:	75cm x 60cm x 30cm weight: 9kg
Deployed size:	200cm x 61cm x 30cm (LxWxH)
3 x Infant Compartments:	48cm x 51cm x 25cm (LxWxH)
Safety tested to:	150kg
Working Load Limit (WLL):	100kg
Material Construction:	Heavy duty Flame Retardant PVC, 33H foam padding, P300 plastic base, 50mm Hi Viz Yellow soft touch pulling webbings, Heavy duty 10mm white mesh.

Conforming Standard:

CE CA 015788 (MHRA)



EvacKRESH



Using the EvacKRESH

- In an emergency, the EvacKRESH is quickly removed from its wall mounted storage bag.
- The infants can then be transferred into the EvacKRESH compartments and placed under the mesh cover.
- Place the infants with their head facing the opening in the compartment
- Place as many infants in each compartment as their size will allow.
- Once loaded slide the EvacKRESH to safety in the direction of the printed arrows using the pulling straps at either end.
- If going up or down stairs always keep the direction arrows pointing up the stairs.
- Constantly monitor the position of the infants in the compartments, if necessary stop and quickly reposition them if they are in distress.



Testing & Cleaning

The EvacKRESH should be inspected regularly for any signs of damage or excessive wear and replaced if necessary. If wiped or washed clean, any detergents or disinfectants should be well rinsed off and the EvacKRESH dried before being replaced in its wall mounted storage bag.

Further Information :



Contact – alison@3et.co.uk

Tel: (+44) 07585 541991

CE CA 015788 (MHRA)