



Product description

The EvacPRO is a flexible lightweight compact stretcher system for use by the emergency services, to evacuate immobile people from such places as nursing homes, hospitals, airplanes, trains, pubs, clubs, schools and hotels in the event of a medical emergency or other emergency situation where the use of conventional patient transfer equipment would not work.

The EvacPRO is designed to facilitate the manual movement of immobile patients by maximising the use of sliding techniques where possible and minimising the impact on the personnel performing the movement through the use of good ergonomic design features.

The EvacPRO is designed to stabilise and hold the patient securely whilst being moved but still have a good degree of flexibility to negotiate obstacles along the transfer route. If required extra stability and support can be added by inserting a standard medical long board into the long board pocket on the upper surface of the EvacPRO. This can be performed either with or without the patient in the EvacPRO, from either the foot or head end via the Velcro openings in the pocket.

Specifications

Packed size:	55cm x 30cm x 30cm	weight: 11kg
Deployed size:	206cm x 152cm x 2cm	
Safety tested to:	500kg	

Working Load Limit (WLL):

400kg

Material Construction:

P300 Plastic base (black 2mm board is replaceable), heavy duty PVC upper, 50mm colour coded safety webbings fitted with metal seatbelt type adjustable buckles.

Conforming Standard:

CE CA 015788 (MHRA)



EvacPRO



Using the EvacPRO

- When required, take the EvacPRO to the patient, undo the 2 securing webbing straps and holding the carry handle allow the EvacPRO to fall and un roll out of the foot pocket. Undo all the securing webbing straps and if required insert the medical long board.
- The patient can then be transferred onto the EvacPRO from either bed / floor/ chair.
- Place their head on the marked "HEAD" pad and if available pad the patient with pillows and blankets.
- Secure the patient in place on the EvacPRO using the 4 x cross straps and then the 2 diagonal straps.
- The length of the EvacPRO can then be adjusted to suit the patient (no long board fitted) by rolling up the foot pocket towards the head end and securing it with the 2 webbing straps fitted with easy adjust cam buckles.
- The patient can then be moved using the pulling ladder handle straps at either end of the EvacPRO and the side ladder handles as required. Additional red webbing lifting handles are positioned along the length of the EvacPRO and at either end there are yellow heavy duty anchor loops to be used as handles or for attaching safety lines to assist in the safe movement of the EvacPRO up or down a slope. If moving on stairs, always pull the EvacPRO feet first down the stairs.





Testing & Cleaning

The EvacPRO should be inspected regularly for any signs of damage or excessive wear, the black 2mm plastic base can be removed and replaced if damaged, any excessive wear or damage to the main structure should be referred back to the manufacturer. If wiped or washed clean, any detergents or disinfectants should be well rinsed off and the EvacPRO dried before being replaced in its rolled storage position inside the foot pocket.

Further Information :



Contact – alison@3et.co.uk

Tel: (+44) 07585 541991



CA 015788 (MHRA)