



## **Product Description**

The EvacSLING is designed to aid in the evacuation of a small baby from a mother and baby unit. It is designed to carry a fit / well baby keeping the mother hands free to negotiate doors and stairs safely. The EvacSLING is manufactured from wipe clean FR PVC with a Hi Viz yellow shoulder strap, trident buckle adjustment, pouch securing strap and photo florescent strips on both side of baby pouch for easy identification of mother with baby in low light conditions. Wall mounted storage bag.

## **Specifications**

**Packed size:** 70cm x 20cm x 4cm

**Deployed size:** 85cm x 20cm x 20cm

**Safety tested to:** 50kg

**Working Load Limit (WLL):** 20kg

**Material Construction:** FR PVC, Polyester webbing, Acetal plastic buckles

**Conforming Standard:**  **CA 015788 (MHRA)**



## **Using the EvacSLING**

- In an emergency, the EvacSLING is quickly removed from its wall mounted storage bag.
- The EvacSLING is placed over the mother's head onto one shoulder with the pouch positioned diagonally across the chest, adjust the shoulder strap to fit.
- Open the small trident clip closing the pouch opening and place the child in the EvacSLING.
- Secure the child in the EvacSLING with the small trident clip and adjust to close the pouch securing the child
- The mother can now make her way from the building with her child secure in the EvacSLING and hands free to negotiate doors and stairs.

## **Testing & Cleaning**

The EvacSLING should be inspected regularly for any signs of damage or excessive wear and replaced if necessary. If wiped or washed clean, any detergents or disinfectants should be well rinsed off and the EvacSLING dried before being replaced in its wall mounted storage bag.

## **Further Information :**