

## Hair Samples for DNA Testing by Weatherbys

The CBHS uses Weatherbys for all pure and part-bred Cleveland Bay DNA testing.

Hair samples should be collected by a vet at the time of completion of the registration and ID chart.

The hair sample should be **pulled** (scissors should not be used) and include the root of the hairs.


Ensure clean hands are used, and place the sample into a clean, sealable, plastic bag.

Please do not puncture bag with a staple, **affix label with adhesive tape.**

The label below may be printed and used, or otherwise, please ensure that you attach the information requested on the label below to the outside of the sealed plastic bag containing the hair.

**The hair will not be removed from the bag at any stage prior to arriving with Weatherbys for testing.**

Thank you for your assistance. This will help Weatherbys ensure the process is efficient.

	<p>Name of Horse: _____</p> <p>Colt/Filly: _____ DOB: _____</p> <p>Affix Spare Microchip Label Below</p>
---	--