



Digital Workflow Instructions

Submitting Digital Cases to Summit Horizon Dental Lab

Submitting a digital case should be simple. Follow the steps below for your scanner system. If you ever need assistance, our team is here to help.

3Shape TRIOS

How to Send a Case to Summit Horizon Dental Lab

1. Open the 3Shape Communicate portal.
2. Complete scan (prep arch, opposing, bite).
3. Enter patient details and restoration type.
4. In the Lab Search bar, type:
 "Summit Horizon Dental Lab" (St. Louis, MO).
5. Add RX notes and shade information.
6. Submit case.

Confirmation: You'll receive a digital receipt, and our team will verify case details.

Quick Reminder

Always attach your RX and include a shade photo when available.
This small step saves time and ensures esthetic accuracy.