



Digital Workflow Instructions

Submitting Digital Cases to Summit Horizon Dental Lab

Submitting a digital case should be simple. Follow the steps below for your scanner system. If you ever need assistance, our team is here to help.

Medit Scanner

Sending Your Case to Summit Horizon Dental Lab

1. Open Medit Link software.
2. Scan arches, bite, and soft tissue as appropriate.
3. Add case information (patient ID, restoration type, notes).
4. Select **Lab > Search** then enter:
 "Summit Horizon Dental Lab" (St. Louis, MO).
5. Attach RX and photos.
6. Submit to the lab.

Confirmation: Summit Horizon will acknowledge and confirm receipt.

Quick Reminder

Always attach your RX and include a shade photo when available.
This small step saves time and ensures esthetic accuracy.