



Digital Workflow Instructions

Submitting Digital Cases to Summit Horizon Dental Lab

Submitting a digital case should be simple. Follow the steps below for your scanner system. If you ever need assistance, our team is here to help.

iTero Scanner

How to Select Summit Horizon Dental Lab in iTero Portal

1. Log in to your iTero scanner.
2. Capture full arch scan(s), bite, and opposing as needed.
3. Complete case details: patient name/ID, restoration type, and shade.
4. In the **Lab Selection** field, search for:
 "Summit Horizon Dental Lab" (St. Louis, MO).
5. Attach your digital RX form and photos (if available).
6. Submit the case.

Confirmation: Summit Horizon will review and contact you to confirm details.

Quick Reminder

Always attach your RX and include a shade photo when available.
This small step saves time and ensures esthetic accuracy.