

QUICKSTART GUIDE


PEROXigard® Ready to Use



CONTACT TIME		
DAILY USE	BACTERICIDAL FUNGICIDAL TUBERCULOCIDAL	VIRUCIDAL (Parvovirus)
1 Minute*		
Surface should be visibly wet for the entire contact time.		

Peroxigard® Ready to Use is excellent for daily use and outbreak situations. For more information on a specific organism, visit the Additional Claims section on the Peroxigard Ready to Use page at PeroxigardDisinfectants.com

How to Use Peroxigard Ready to Use

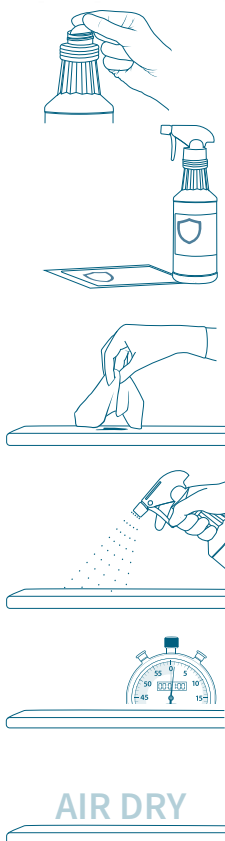

STEP 1

Prepare the bottle

Remove the lid and seal. Use approved sprayers and foamers to make dispensing onto a surface easy and for even distribution.

STEP 2

Apply a workplace label to any secondary bottles

Use a Peroxigard Ready to Use workplace label and record the lot number and the date the product was decanted.

STEP 3

Pre-clean heavily soiled areas

Remove visible soil and pre-clean the area by working from the least to the most contaminated area.

STEP 4

Apply Peroxigard Ready to Use onto the surface

The entire surface area should be covered and appear visibly wet. Wettability and surface coverage are equally as important to ensure disinfection can be achieved in one application.

STEP 5

Allow surface to remain visibly wet for 1 minute*

Allowing surfaces to remain wet for 1 minute* kills bacteria, viruses and fungi.

STEP 6

Allow to air dry

If desired, you may wipe excess solution using a clean cloth after the required contact time.

Success Tips

Spray and Walk Away.

Best practice is to simply apply the product and walk away, allowing to air dry.

Wait Before Wiping Dry.

Always follow the required contact time when disinfecting a surface. If you wish to wipe excess solution using a clean cloth, ensure you wait until after the required contact time has been achieved.

Switch to Foaming.

Peroxigard is non-irritating to eyes and skin and is not harmful to inhale. The aerosolization of Peroxigard however may cause sensitive individuals to cough. Try using a trigger foamer to help reduce aerosolization and improve surface coverage.

Always Ready-to-Use.

The solution does not require dilution or mixing with water, making it simple and saving you time. Simply attach a trigger spray or foamer to the bottle or secondary container and you are ready to disinfect.



Peroxigard is a member of the Virox family of brands.

DISCLAIMER: You must read and follow the instructions on the label for our product and use the product for the particular applications specified on the product label. To the full extent permitted under applicable law, Virox Technologies Inc. shall not be liable for any damages whatsoever, arising out of or otherwise in connection with any failure to follow those instructions or any use of its products which is not specified on the product's label. The instructions on the product label are not a warranty, express or implied, and Virox Technologies, Inc. expressly disclaims any implied warranties, including any warranty of fitness for a particular purpose. Under applicable law, some of the above waivers or disclaimers of warranty, or exclusions of liability for certain types of damages may not apply to you. In such cases, our liability will be limited to the fullest extent permitted by applicable law. Product shots have been digitally altered and are not accurate as shown. *See reference sheet for a complete list of efficacy claims, additional features and directions for use.

Accelerated Hydrogen Peroxide® and Design are trademarks of Diversey, Inc.

B-081825-8492-03F

© 2025 Virox Technologies Inc.