


Dochek Amazewell Drug Test Cup Procedure Card

TEST PROCEDURE

Test should be performed at room temperature (59°F- 86°F / 15°C - 30°C).

1. After the urine has been collected, place the test cup on a flat surface.
2. Read the temperature strip on the cup immediately
3. Start the timer. Peel the label from right to left. Read the result at 5 minutes. The result can be stable for 60 minutes. Do not read the result after 60 minutes.

Note: Results after more than 60 minutes may be not accurate and should not be read.

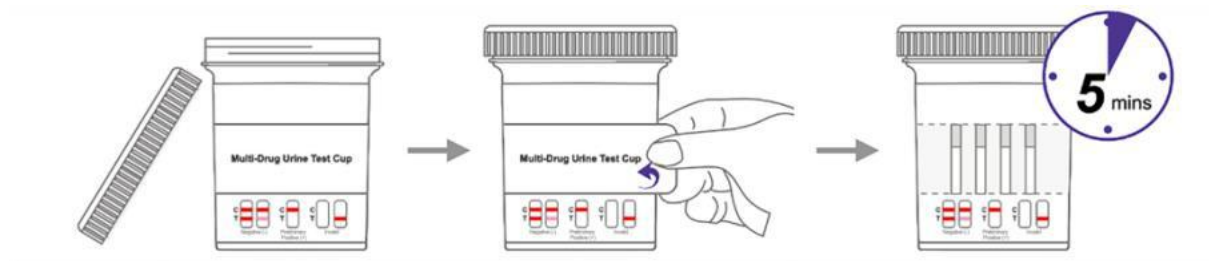

INTERPRETATION OF TEST RESULTS

Preliminary positive (+)

A color band is visible in each control region (C). If no color band appears in the appropriate drug test region(T), a preliminary positive result is indicated for the corresponding drug of that specific test region.

Negative (-)

If a color band is visible in each control region (C) and the appropriate drug test region(T), it indicates that the concentration of the corresponding drug of that specific test region is absent or below the detection limit of the test.

Invalid

If a color band is not visible in the control region(C), the test is invalid. Another test should be run to re-evaluate the specimen. If test still fails, please contact the distributor or the store, where you bought the product, with the lot number.

NOTE: There is no meaning attributed to line color intensity or width. Any visible line is considered to be a line.

