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INSTRUCTIONS FOR USE: READ CAREFULLY!

BSS DELTAMEDICA

Sterile eye irrigation solution (Balanced Salt Solution)

Composition

1000 ml contain:

Sodium chloride

6.40 g

Potassium chloride

0.75 g

Calcium chloride x 2 H₂O

0.48 g

Magnesium chloride x 6 H₂O

0.30 g

Sodium acetate x 3 H₂O

3.90 g

Trisodium citrate x 2 H₂O

1.70 g

Water for injection

up to 1000 ml

Elektrolyte (mmol/l)

Na⁺155,50

Ca⁺⁺3,27

Cl⁻129,00

Citrat³⁻5,78

Osmolarity

300 mosm/l

pH value

6,8 – 7,4

Pharmaceutical forms and pack sizes

BSS DELTAMEDICA is available in packs of

10 bottles, each

250 ml (plastic (PP))

10 bottles, each

500 ml (glass)

20 bottles, each

500 ml (plastic (PP))

DELTAMEDICA GmbH

Ernst-Wagner-Weg 1-5

72766 Reutlingen

Germany

Tel.: +49 (0) 7121 - 99 21-15

Fax: +49 (0) 7121 - 99 21-31

info@deltamedica.de

www.deltamedica.de

BSS DELTAMEEDICA a sterile physiological salt solution for ophthalmological use.

Indications

- For irrigation of the frontal chamber during cataract surgery and other intraocular interventions
- For rinsing out alpha-chymotrypsine after using enzymatic zonulolysis
- As an intraocular irrigation of the frontal ocular chamber in ocular chamber haemorrhages
- For irrigation of the cornea during surgery for strabismus and other extraocular interventions
- For irrigation of the conjunctiva after the installation of fluoresceine
- For keeping the cornea and conjunctiva moist during laser treatment

Contraindications, side effects and interactions

Contraindications, side effects and interactions with other substances are not known.

Dosage

In the case of intraocular irrigations, the volume to be used varies, depending on the individual and the type of operation, and must be determined by the doctor in each case regardless.

Duration of use

According to the specific needs.

Notes

- Not for intravenous infusion.
- Not to be taken orally.
- The content of the bottle should be clear, and the container undamaged.
- Single use only. Do not re-sterilise.
- After the container has been opened, BSS DELTAMEDICA must be used immediately. Remaining amounts must be discarded. If re-used, infections with severe complications or even death may occur because of contaminations.
- Store at 4°C - 25 °C.
- The expiry date is printed on the labels in the bottle and the box. BSS DELTAMEDICA must not be used after this date.
- There are no reasons against using BSS DELTAMEDICA during pregnancy and breast-feeding, since BSS DELTAMEDICA is a physiological salt solution.

Notes regarding use

- A transfer set suitable for the application must be used.
- Follow the instructions for the use of the transfer set used.
- Attach the transfer set to the bottle as follows:
 - Remove the cap and disinfect the intended puncture site with an alcohol swab.
 - Pierce the tip of the transfusion set through the septum. Ensure aseptic operation.
 - The liquid should flow before use to prevent air bubbles in the tube from entering the eye.
 - Proceed as described in the instructions for use of the transfer set.
- The entry location on the septum is intended for single use only.

Date of last revision: 05/2024

Store eye irrigation solution out of the reach of children!

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Valor pH

6,8 – 7,4

Farmaceutical forms and pack sizes

BSS DELTAMEDICA is available in packs of

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