NaCl 0,9% SPÜLLÖSUNG DELTAMEDICA

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NaCl 0,9% IRRIGATION SOLUTION DELTAMEDICA

INSTRUCTIONS FOR USE: READ CAREFULLY!

PVC-free bag containing sterile NaCl 0.9% Irrigation Solution

Composition

1000 ml contains:

Sodium chloride	9 g
Water for injection	up to 1000 ml
Electrolytes	
Na ⁺	154 mmol/l
Cl ⁻	154 mmol/l
Osmolarity	309 mosm/l
pH value	4,5 - 7,0

Pharmaceutical forms and pack sizes

NaCl 0.9% Irrigation Solution DELTAMEDICA is available in packs of

4 bags, each	3000 ml
2 bags, each	5000 ml

DELTAMEDICA GmbH

Ernst-Wagner-Weg 1-5
D-72766 Reutlingen

Tel. +49 (0) 7121/99 21-15
Fax +49 (0) 7121/99 21-31

info@deltamedica.de
www.deltamedica.de

Indications

- For irrigation during urological operations and for washing out catheters.
- Irrigation solution for tumescence anaesthesia during liposuction.
- For intraoperative irrigation.
- For wound irrigation.
- For irrigation in endoscopic examinations in body cavities.
- For intraoperative irrigation in arthroscopies with mechanical instruments.

Contraindications

Not to be used for electrosurgery.
Do not use for irrigation in open heart surgery.

Dosage

In the amount required for the treatment.

Duration of the application

According to the particular requirement.

Side effects

The absorption of large amounts of irrigation solution through a perforation or open wounds can lead to strain on the cardiovascular system, cardiac failure or disorders of the electrolyte metabolism. This can be a particular problem in the case of patients with existing kidney or cardiopulmonary disease.

Particular care is indicated in the monitoring of patients with limited kidney function, as in such cases the absorption of significant amounts of irrigation solution can lead to water intoxication syndrome.

Notes

- Not for intravenous infusion.
- Not to be taken orally.
- To be used only if the solution is clear and the bag undamaged.
- Single use only. Do not re-sterilise.
- After the container has been opened the NaCl 0.9% Irrigation solution DELTAMEDICA must be used immediately. Remaining amounts must be discarded. If re-used, infections with severe complications or even death may occur because of contaminations.
- Store at 4°C - 25°C.
- The expiry date is printed on the bag and the box label. The NaCl 0.9% Irrigation solution DELTAMEDICA must not be used after this date.
- Do not heat the bag above 65°C.
- Do not use a microwave device to heat the solution.

Instructions for use

- A transfer set suitable for the application must be used.
- Follow the instructions for use of the respective transfer set.
- Attach the transfer set to the bag as follows:
 1. Hang up the bag using the punched hole provided.
 2. Remove the protective cap from the transfer set.
 3. Close the regulator clamp of the transfer set and push the set connector firmly into the plug, or attach the Luer lock. Whilst doing this take care to maintain aseptic working procedures.
 4. If the Irri lock of the irrigation bag is connected to the Luer lock of the transfer set, the next step is to break the break connector of the Irri lock on the bag.
 5. Continue as described in the instructions for transfer set.

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