RINGER SPÜLLÖSUNG DELTAMEDICA

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RINGER IRRIGATION SOLUTION DELTAMEDICA

INSTRUCTIONS FOR USE: READ CAREFULLY!

PVC-free bag containing sterile Ringer Irrigation Solution.

Composition

1000 ml contains:	
Sodium chloride	8.60 g
Potassium chloride	0.30 g
Calcium chloride x 2 H ₂ O	0.33 g
Water for injection	up to 1000 ml
Electrolytes	
Na ⁺	147 mmol/l
K ⁺	4 mmol/l
Ca ⁺⁺	2.25 mmol/l
Cl ⁻	155.5 mmol/l

Pharmaceutical forms and pack sizes

Ringer Irrigation solution DELTAMEDICA is available in packs of	
4 bags, each	3000 ml
2 bags, each	5000 ml

DELTAMEDICA GmbH

Ernst-Wagner-Weg 1-5
D-72766 Reutlingen

Tel. +49 (0) 7121/99 21-15
Fax +49 (0) 7121/99 21-31

info@deltamedita.de
www.deltamedica.de

Indications

- For irrigation and cleansing during surgical interventions
- For irrigation of wounds and burns
- For irrigation during endoscopic investigative surgery in body cavities
- For intraoperative irrigation in arthroscopies with mechanical instruments
- For postoperative irrigation

Contraindications

Not to be used for irrigation during electrosurgery.

Dosage

In the amount required for the treatment.

Duration of application

According to the particular requirement.

Side effects

No side effects are known where the product is used as intended.

Overdosage

In case of overdose within the framework of wound irrigation and intraoperative use, the irrigation solution may be absorbed, leading to a hypervolemia in the sense of a hypotonic hyperhydration (hyponatremia).

Symptoms of hyperhydration include headaches, urge to vomit, anxiety and disorientation. A coma-like state may be observed in severe cases.

Corrective measures in the event of an overdose
Stop irrigation, correct the water and electrolyte household in response to the respective clinical situation.

- Not for intravenous infusion.
- Not to be taken orally.
- To be used only if the solution is clear and the container undamaged.
- Single use only. Do not re-sterilise.
- After the container has been opened, Ringer Irrigation solution DELTAMEDICA must be used immediately. Remaining amounts must be discarded. If re-used, infections with severe complications or even death may occur because of contaminations.
- Store at 4°C - 25°C.
- The expiry date is printed on the bag and the box label. The Ringer Irrigation solution DELTAMEDICA must not be used after this date.
- Do not heat the bag above 65°C.
- Do not use a microwave device to heat the solution.

Instructions for use

- A transfer set suitable for the application must be used.
- Follow the instructions for use of the respective transfer set.
- Attach the transfer set to the bag as follows:
 1. Hang up the bag using the punched hole provided.
 2. Remove the protective cap from the transfer set.
 3. Close the regulator clamp of the transfer set and push the set connector firmly into the plug or attach the Luer lock. Whilst doing this take care to maintain aseptic working procedures.
 4. If the Irri lock of the irrigation bag is connected to the Luer lock of the transfer set, the next step is to break the break connector of the Irri lock on the bag
 5. Continue as described in the instructions for transfer set.

Notes

Date of last revision 02/2021

Store irrigation solution out of the reach of children!

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