Triumph Essentials Rollator

Item#: 700-1500

Instruction Booklet
Livret d'instructions

TRIUMPH
essentials

Quick Start Guide

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Introduction

Thank you for your purchase of a Triumph Essentials Rollator.

Please ensure that all users review the instructions carefully prior to use and are familiar with safety and operating instructions. Before assembling and using your new Triumph Essentials Rollator, please check that no damage to the frame or parts has occurred during shipping.

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For your safety

Please read these important tips to ensure your safety and get the most out of your rollator.

When using your rollator

1. When using the rollator, ensure that the rollator is fully opened and side frames are locked in position.
2. Always confirm that the handgrips are at equal height, and height adjustment levers are in the closed position.

Care and maintenance

1. Do not use on stairs or escalators.
2. Never use the rollator in a moving vehicle

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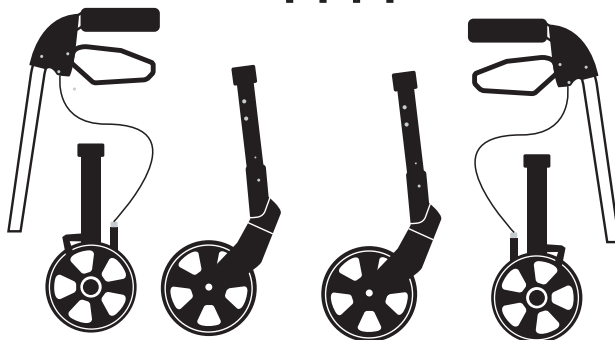
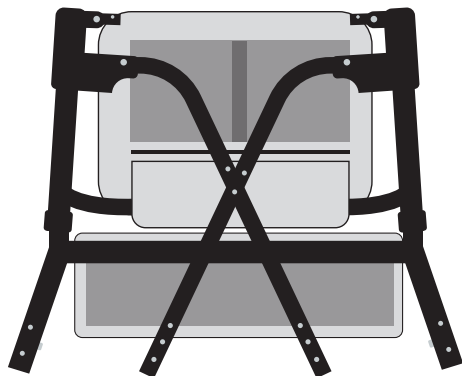
Instructions
de montage

Open frame

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Install leg tubes to desired height

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Make sure the front leg tube with fork goes on the front, and the rear wheel leg tube is installed on the back with the wheel on the outside of the frame.

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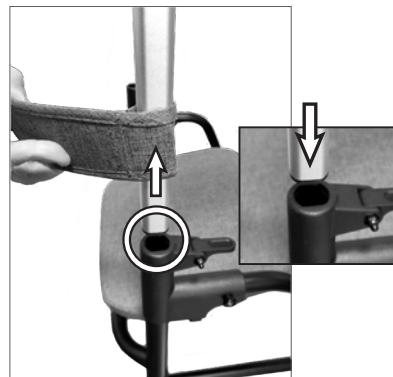
Folding & Storage

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To fold the rollator

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Lift seat & press folding release buttons.

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Squeeze the brakes to slow down. Push down on the brake handles to lock the brakes. Pull up on the brake handles to unlock the brakes.

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Adjustment of brakes / Ajustement des freins

If the brake is too loose, loosen the locking nut, and turn the adjustment screw counter-clockwise.

If the brake is too tight, loosen the locking nut, and turn the adjustment screw clockwise.

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Stand up as straight as possible and walk with the rollator close to your body. Keep your feet centred. Do not push the rollator away from your body, as this will cause you to bend your back, and could potentially cause a fall.

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To sit on the rollator, ensure that the brakes are in the LOCKED position, and the rollator does not move. Using the handles for stability, carefully turn around. Gently back up until the back of your legs are touching the edge of the seat, and sit down. Ensure both feet make contact with the ground while seated.

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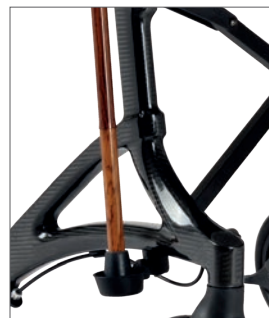
Maintenance / Entretien

Regular maintenance will preserve the life and safety of your rollator.
The following should be checked at regular intervals.

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To adjust brake tension: If the brake handle is too tight, loosen the locking nut, and turn the adjustment screw clockwise. Re-tighten the locking nut. If the brake handle is too loose, loosen the locking nut, and turn the adjustment screw counterclockwise. Re-tighten the locking nut.

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Warranty / Garantie

Triumph covers the Essentials rollator with a two (2) year warranty on the frame. All other components are covered for one (1) year against defects in material and workmanship.

This warranty is valid for the original purchaser only and is not transferable. Triumph Mobility will not warranty any parts that have been damaged due to improper installation, modification, and / or misuse.

For warranty service, please return your rollator to your local dealer . Service charges are of the discretion of the dealer and are not the responsibility of Triumph Mobility.

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Read instructions carefully before use
Triumph Essentials Rollator is designed for indoor/outdoor use
Instruction manual must be given to the user

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Triumph Mobility
151 Bentley Ave., Unit 4, Nepean, ON, K2E6T7
info@triumphmobility.com
1.855.546.0711

This manual is subject to updates and changes without notice.
Please check www.triumphmobility.com for most current version of this document.

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