


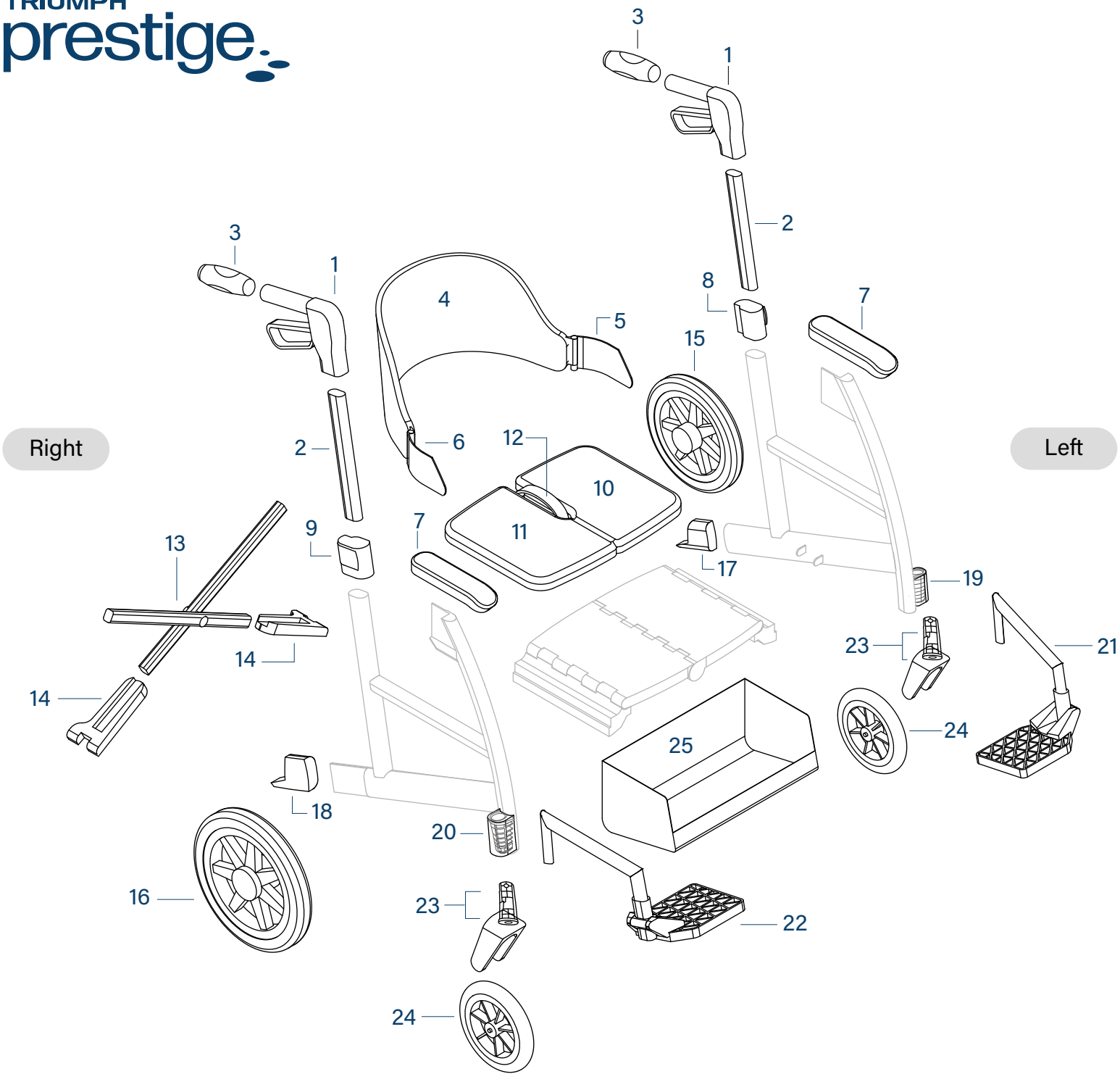

Callout	Part#	Description	UoM
1	600-1000	Brake housing - Complete	EA
2	600-1005	Handle tube Complete	EA
3	600-1010	Handle grip (PR)	PR
4	600-1020	Backrest complete with hinge (21" Seat)	EA
	600-1021	Backrest complete with hinge (19" Seat)	EA
	600-1022	Backrest only	EA
5	600-1023	Backrest hinge (21" Seat)- Left	EA
	600-1025	Backrest hinge (19" Seat)- Left	EA
6	600-1024	Backrest hinge (21" Seat) - Right	EA
	600-1026	Backrest hinge (19" Seat) - Right	EA
7	600-1030	Armrest (w. screws)	EA
8	600-1035	Handle height adjustment housing - Left	EA
9	600-1036	Handle height adjustment housing - Right	EA
10	600-1040	Seat cushion - Left	EA
11	600-1045	Seat cushion - Right	EA
12	600-1050	Handle strap	EA
13	600-1055	Cross bar brace assembly	EA
14	600-1060	Cross bar bracket	EA
15	600-1065	Rear wheel - Left	EA
16	600-1070	Rear wheel - Right	EA
17	600-1075	Curb climber - Left	EA
18	600-1080	Curb climber - Right	EA
19	600-1085	Footrest pivot insert - Left	EA
20	600-1090	Footrest pivot insert - Right	EA
21	600-1095	Footrest Complete - Left	EA
22	600-1100	Footrest Complete - Right	EA
	600-1105	Footrest Complete (PAIR)	PR
23	600-1115	Front fork & wheel mount	EA
24	600-1120	Front wheel & axle bolt	EA
25	600-3000	Shopping bag (underseat)	EA