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Instructions for use

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Orthopedic seat Beanseat  
Stabilo System

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Thank you for choosing our product. The NOEL Żebrowski sp. j. has designed and manufactured orthopedic seat Beanseat Stabulo System to the highest safety and functional use standards. We are confident that our product will meet your highest requirements and expectations. These instructions for use are an integral part of vacuum positioning system stabilizing torso, limbs and head orthopedic seat Beanseat Stabulo System. Please read the instructions thoroughly before using the product.

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### Vacuum positioning system stabilizing torso, limbs and head orthopedic seat Beanseat Stabulo System

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Product designed for children and adults with cerebral palsy, compensating for and alleviating the disability by stabilizing the patient in sitting or lying position.

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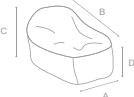
#### INDICATIONS

Orthopedic seat Beanseat Stabulo System can be used for:

- alleviating the effects of remaining in one position for extended periods (e.g. contractures)
- stabilizing patient with muscular flaccidity

Beanseat, Beanseat Velcro	Typ Type	Wymiary [cm] Dimensions [cm]			
		A	B	C	D
	<b>M</b>	60	80	50	40
	<b>L</b>	70	100	70	50
	<b>XL</b>	70	100	100	70

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#### PRODUCT SAFETY WARNINGS

- long endured of patient in one position can lead to bedsores
- use of airy cover is advisable if increased ventilation is required around the body
- avoid contact with sharp objects that may damage the surface
- do not clean using solvent-based cleaners, e.g. toluene or acetone
- do not pull on valve
- do not overinflate seat
- use only special pump included with seat for inflation
- do not iron
- in case of tear, microgranules may escape which may be dangerous for small children
- only persons who received instructions or training may use the device

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## ATTENTION

- the thickness of the cushion can naturally reduce by 5% in the initial period of use
- in case of considerable changes of temperature use pump again to obtain proper hardness (higher temperature softens the cushion, lower temperature hardens the cushion)

## INSTRUCTIONS FOR USE ELECTRIC PUMP

1. Take out tube from box with electric pump.
2. Screw off valve cap, screw tube (tapped part) into cushion valve, and push second tube into SOFT socket in electric pump. Turn on pump and wait until cushion will be soft enough, then turn pump off.
3. Start shaping cushion (either manually to adjust shape or by sitting or leaning on it).
4. Change position of tube from SOFT to HARD and turn pump on removing air until the desired hardness has been reached.
5. Screw off tube, put cap back on and tighten.
6. To change the shape repeat steps from 2 to 5.

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## MAINTENANCE

- regularly check condition of product – pay attention to any cracks and rips
- screw on valve cap before washing
- hand wash seat under running water using a standard detergent e.g. a washing up liquid
- suitable for washing in washing machine at 40°C cycle. Fabric softener is sufficient, does not require use of washing powder
- wipe dry with paper towel
- standard disinfectants used in hospitals can be used
- do not use solvent-based cleaners such as toluene or acetone

## WARRANTY

- the warranty is provided for a period of 24 months
- the warranty does not cover mechanical damage to orthopedic seat Beanseat Stablio System
- if in doubt, please contact seller or manufacturer directly
- report any incidents to seller or manufacturer directly

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## DECLARATION OF CONFORMITY

Vacuum positioning system stabilizing torso, limbs and head orthopedic seat Beanseat Stabilo System medical device complies with the basic requirements of the Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices.

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