Triumph Escape HD

Item#: 500-30217

Instruction Booklet
Livret d'instructions

TRIUMPH
escapeTM
HD

Quick Start Guide

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Introduction

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Pull the red knob below the seat to release the frame lock. Pull/push handles apart, and push down on the seat.

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Installing the backstrap Installation du dossier souple

The backstrap is height and angle adjustable.

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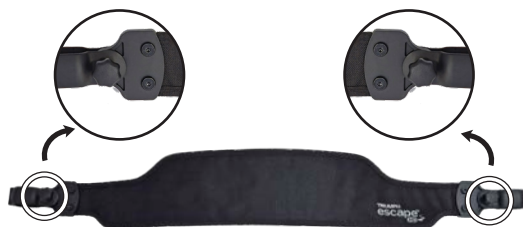
Installing the backstrap Installation du dossier souple

Loosen the knob to open backstrap clamp

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Set the desired height and angle of the backstrap

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Set the desired height and angle of the backstrap either above or below the brake cable and tighten the knob.

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Note: Backrest clamp must be removed to adjust backstrap above or below the brake cable.

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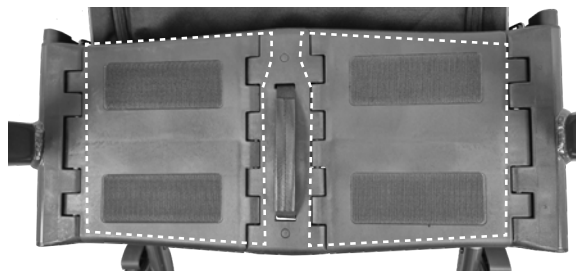
Align the front edge of the seat cushion with the front edge of the seat.

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Position the left straight edge along the left side of the seat (or the right straight edge along the right side of the seat).

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Installing the shopping bag Installation du sac de magasinage

1. Attach the top hooks

Hang the two top hooks of the shopping bag onto the designated bag hooks located under the seat frame.

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2. Secure the bottom straps

Wrap the Velcro straps around the lower frame tube and fasten them through the loops to ensure the bag is securely attached.

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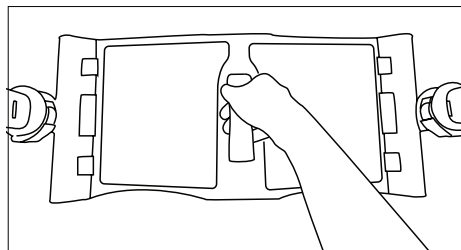
To fold rollator, pull up on the handle strap located in the centre of the seat.

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Frame Lock (ON/OFF) Verrouillage de trame (ON / OFF)

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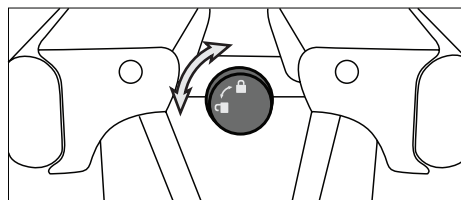
Rotate the red knob 90degrees. This will deactivate the frame lock.


The rollator will still fold normally, but will not lock in the folded position. To re-enable the frame lock feature, rotate the knob 90 degrees.

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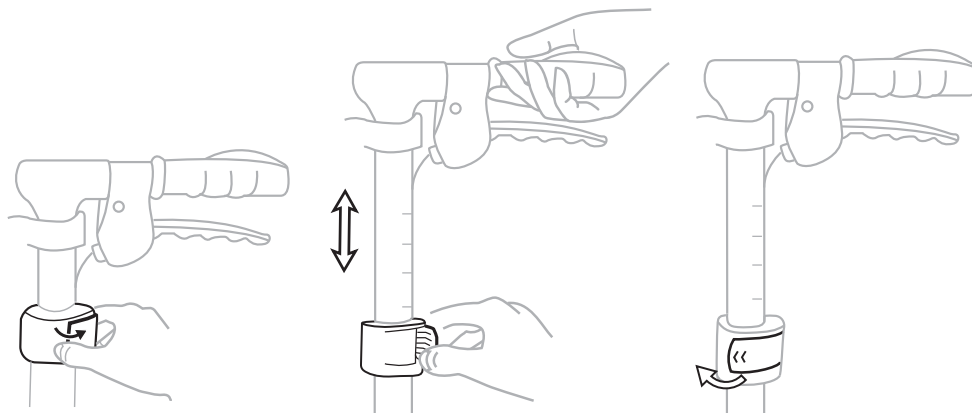Lock the brake by pushing down on the handle. The handle will lock into a downward position when properly engaged.

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Release the brake by pulling up on the handle.

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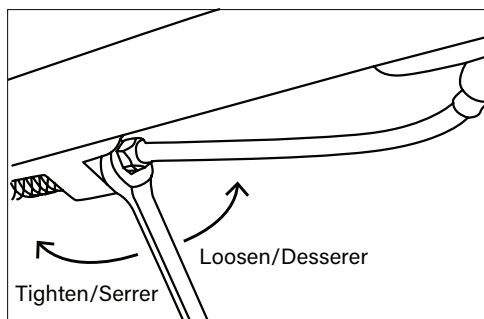
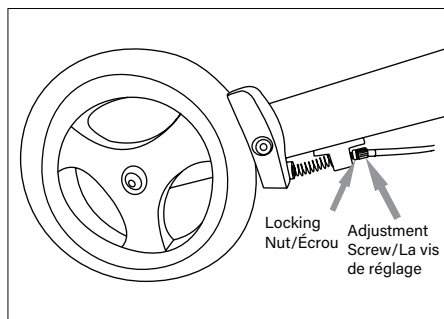
Control speed by squeezing the handle.

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
Adjustment of brakes / Ajustement des freins

If the brake is too loose, loosen the locking nut, and turn the adjustment screw counter-clockwise. If the brake is too tight, loosen the locking nut, and turn the adjustment screw clockwise.

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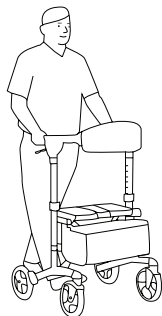
Stand up as straight as possible, and walk with the rollator close to your body. Keep your feet centred. Do not push the rollator away from your body, as this will cause you to bend your back, and could cause a fall.

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To sit on the rollator, ensure that the brakes are in the LOCKED position, and the rollator does not move. Using the handles for stability, carefully turn around. Gently back up until the back of your legs are touching the edge of the seat, and sit down. Ensure both feet make contact with the ground while seated.

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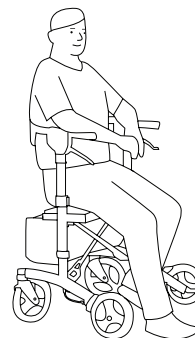
Use the curb climber to overcome obstacles such as curbs and thresholds. To use the curb climber, keep both hands on the handles, and place foot onto the curb climber to lift the front wheels of the rollator.

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
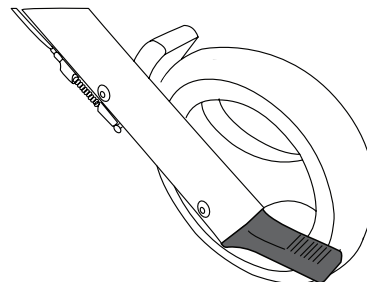
To adjust brake tension: If the brake handle is too tight, loosen the locking nut, and turn the adjustment screw clockwise. Re-tighten the locking nut. If the brake handle is too loose, loosen the locking nut, and turn the adjustment screw counterclockwise. Re-tighten the locking nut.

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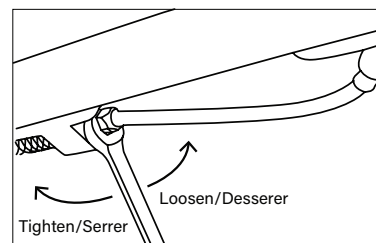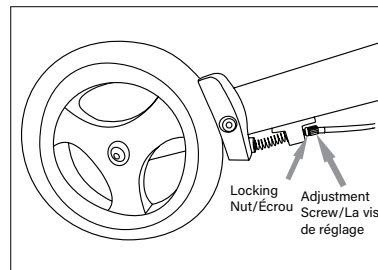Warranty / Garantie

Triumph Mobility covers the Escape Rollator by a lifetime warranty on the frame, and a five (5) year warranty on the brake handles against defects in materials and workmanship. All other components are warranted for two (2) year.

This warranty is valid for the original purchaser only and is not transferable. Triumph Mobility will not warranty any parts that have been damaged due to improper installation, modification, and / or misuse.

For warranty service, please return your rollator to your local dealer . Service charges are at the discretion of the dealer and are not the responsibility of Triumph Mobility.

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Please register your Escape rollator online at **www.triumphmobility.com/product-registration**. When registering, please include the serial number. The serial number can be located on the right side of the frame.

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Theft & Vandalism Insurance / Assurance vol et vandalisme

Every Escape rollator comes with automatic insurance for your peace of mind. For a \$100 deductible (plus shipping and applicable taxes), we will take care of getting a replacement to your doorstep. This insurance covers the first two years that you own your rollator (like most guarantees, it is not transferable if the rollator changes ownership) To make a claim, we ask our clients to visit:

www.triumphmobility.com/product-insurance to provide us with a copy of the relevant police report as well as their rollator's serial number and proof of purchase. We'll do the rest. Let us help you get mobile again!

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Read Instructions carefully before use. Rollator is designed for indoor & outdoor use. Do NOT use as a wheelchair or as a transport chair. Instruction manual must be given to the user.

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info@triumphmobility.com

1.855.546.0711

This manual is subject to updates and changes without notice. Please check www.triumphmobility.com for most current version of this document.

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