

TruPlan

v3.4.0

(0442)

Release Notes

What's New in Version 3.4.0

Below is a summary of the new features that are introduced in version 3.4.0 of TruPlan. For a full description of product functionality please refer to the user manual.

New image and report exporting options

- Various image exporting options added to enhance case review workflows:
 - Export cropped MPR views of the landing zone (cross-section and long-axis orientations)
 - Export rotational batch of secondary captures from volume-rendered view
 - Rotate about vertical, horizontal, or LAA axis
 - Export all report images as a series of DICOM secondary captures
- Exported images can be viewed along with the report in cvi42 | Web Viewer for easy access away from the workstation

Study tagging

- Add/remove study tags for improved worklist organization
- Search and filter studies by tag to easily find cases

Fluoro module

- Updated Fossa Ovalis hinge point initialization to Inferior position to optimize trans-septal planning

Resolved Issues

Number	Summary
HV-2315	DICOM parsing exception causes TruPlan to close when selecting specific study in the study list
HV-2261	Overwriting auto-defined bookmarks in TEE module is not saved

Known Issues

Number	Summary
HV-2364	Moving TruPlan window between different size / scaling monitors results in incorrect window scaling <i>Workaround: User can click a different window to cause TruPlan to re-render the window contents with the correct scale, then return to TruPlan</i>
HV-2319	Some series/study list updates are delayed by up to 30 seconds <i>Workaround: Select a different study then select the original study.</i>
HV-2266	Volume viewport does not update when changing rendering mode in some cases <i>Workaround: Drag the 3D volume</i>
HV-1991	Screenshots are not being added to the report in the order they are created <i>Workaround: No workaround</i>
HV-1898	Landing Zone Point, IVC Point, and Septal Crossing Point don't disappear when cine changes phase. <i>Workaround: No workaround</i>
HV-1320	Landing zone cross section always opens in the upper left view <i>Workaround: None</i>
HV-2167	If a user visits the ICE module before Fluoro then the home positions will not be shown <i>Workaround: Click the 'reset' home positions button</i>
HV-2146	Endoluminal view is not saved to a session correctly <i>Workaround: Users can click on one of the viewports and it will update correctly.</i>
HV-1980	Visibility of PDL and Distal LAA measurements not remembered in save session <i>Workaround: Make Distal LAA and PDL not visible again.</i>
HV-1299	Image flickers while taking screen-capture <i>Workaround: No workaround</i>

External Libraries – Copyright and Licensing Requirements

Open-source libraries can be found on the [Circle website](#). It can also be found as Open_Source_Notices.pdf in the install directory of TruPlan.

About licenses

MPEG-4 AVC:

THIS PRODUCT IS LICENSED UNDER THE AVC PATENT PORTFOLIO LICENSE FOR THE PERSONAL USE OF A CONSUMER OR OTHER USES IN WHICH IT DOES NOT RECEIVE REMUNERATION TO (i) ENCODE VIDEO IN COMPLIANCE WITH THE AVC STANDARD ("AVC VIDEO") AND/OR (ii) DECODE AVC VIDEO THAT WAS ENCODED BY A CONSUMER ENGAGED IN A PERSONAL ACTIVITY AND/OR WAS OBTAINED FROM A VIDEO PROVIDER LICENSED TO PROVIDE AVC VIDEO. NO LICENSE IS GRANTED OR SHALL BE IMPLIED FOR ANY OTHER USE. ADDITIONAL INFORMATION MAY BE OBTAINED FROM MPEG LA, L.L.C. SEE [HTTP://WWW.MPEGLA.COM](http://www.mpegla.com)

Support

For further information on installing/exchanging the dynamic libraries please contact Circle Cardiovascular Imaging support.

Phone: +1 (403) 338-1870

E-mail: support@circlecvi.com

Website: circlecvi.com

Circle Cardiovascular Imaging Inc.

Suite 1800, 707 - 8th Ave SW
Calgary, AB, T2P 1H5
Canada