

TruPlan

v3.1.2

(0284)

Release Notes

What's New in Version 3.1.2

Below is a summary of the new features that are introduced in version 3.1.2 of TruPlan. For a full description of product functionality please refer to the user manual.

Addition of a new CT Follow-Up

- A dedicated module for CT follow-up provides tools to easily assess a CT study following the LAAC procedure, including the following tools
 - Peri-device leak (PDL) measurement
 - Distal LAA measurement
 - Pixel thresholding tool
 - Adjustable MIP box to visualize LAAC device
 - Dedicated follow-up report format

Changes to LAA module

- Automation for placement of the landing zone in the ostium of the LAA. User can modify or replace the automated landing zone annotation if desired
- Automated LAA depth marker

Changes to Fluoro module

- Improved rendering of access sheaths with the volume-rendered image
- Improved ability to lock 3D rotation around LAA axis

Changes to Thrombus module

- ROI tool to assess signal intensity and account for noise in the image

Miscellaneous

- Improvements to the volume rendering visualization
- Improved rendering of planning landmarks and measurements in 3D views
- Ability to save a report as DICOM-encapsulated PDF and send to PACS
- Improved thumbnails for Reports, saved sessions, SSO
- Additional parameters included in saved sessions (current module, viewport settings)
- Notification before loading a new study to remind user to save their current session if desired
- Automatic identification of pre- and post- operative studies to open series in the LAA or Follow-Up modules, respectively

System improvements

- Introduction of client/server architecture to better support enterprise deployments
 - One server deployment with a single connection to PACS, centralized user management and workspace storage to support multiple users
- Integration to AD/LDAP server for user authentication
- Data encryption with TLS connection support
- Disk space monitoring to prevent reaching maximum storage capacity

Resolved Issues

Number	Summary
HV-1825	Uninstalling TruPlan is removing the folder C:\Users\<user>\AppData\Roaming\Microsoft\Windows\sendTo
HV-1992	When TruPlan is installed in Windows, different Windows users don't share the list of servers

Known Issues

Number	Summary
	TruPlan will not accept a new license until the current license has expired
HV-1989	<i>Workaround: Current license file can be removed from license folder and TruPlan server restarted, so TruPlan would be in the state of not activated and then it could be re-activated with the new license.</i>

External Libraries – Copyright and Licensing Requirements

This is a list of free and open-source software used in our products, including any modifications made by Circle Cardiovascular Imaging Inc., throughout the course of our development.

Qt v5.15.7

Project Website
License
Download

[Qt Project](#)
[Qt Commercial Enterprise License](#)
[Source Code and Binaries](#)

DCMTK v3.6.7

Project Website
License
Download

[DICOM Toolkit \(DCMTK\) Project](#)
[DCMTK License](#) [DCMJP2K Commercial License](#)
[DCMTK Source Code](#)

Eigen v3.3.7

Project Website
License
Download

[Eigen Project](#)
[Apache License v2.0](#)
[Eigen Source Code](#)

GLM v0.9.9.0

Project Website
Licenses
Download

[OpenGL Mathematics \(GLM\) Project](#)
[The Happy Bunny and MIT Licenses](#)
[GLM v0.9.9.0 Source Code](#)

OpenCV v3.4.6

Project Website
License
Download

[Open Source Computer Vision \(OpenCV\) Project](#)
[OpenCV \(3-Clause BSD\) License](#)
[OpenCV v3.4.6 Source Code](#)

Google Protocol Buffers v3.12.4

Project Website
License
Download

[Protocol Buffers](#)
[Protobuf License](#)
[Source Code and Binaries](#)

ZLIB v1.2.12

Project Website
License
Download

[Zlib Project](#)
[Zlib license](#)
[Source Code and Binaries](#)

Tensorflow v1.13.1

Project Website
License
Download

[Tensorflow Project](#)
[Tensorflow License](#)
[Source Code and Binaries](#)

RLM v15.0BL1

Project Website
License
Download

[Reprise software](#)
Proprietary
[Source Code and Binaries](#)

Boost v1.66

Project Website
License
Download

[Boost Website](#)
[Boost License](#)
[Source Code and Binaries](#)

QtService v2.8.6

Project Website
License
Download

[QtService Website](#)
[QtService License](#)
[Source Code and Binaries](#)

Qtiocompressor v2.3.1

Project Website
License
Download

[Qtiocompressor Website](#)
[Qtiocompressor License](#)
[Source Code and Binaries](#)

QJsonWebToken vbf4d5e8.1.6

Project Website
License
Download

[QJsonWebToken Website](#)
[QJsonWebToken License](#)
[Source Code and Binaries](#)

QCA v2.3.3

Project Website
License
Download

[QCA Website](#)
[QCA License](#)
[Source Code and Binaries](#)

LMDB v0.9.19

Project Website
License
Download

[LMDB Website](#)
[LMDB License](#)
[Source Code and Binaries](#)

ICU v56.1

Project Website
License
Download

[ICU Website](#)
[ICU License](#)
[Source Code and Binaries](#)

OpenLDAP v2.5.7

Project Website
License
Download

[OpenLDAP Website](#)
[OpenLDAP License](#)
[Source Code and Binaries](#)

OpenSSL v1.1.1L

Project Website

[OpenSSL Website](#)

License
Download

[OpenSSL License](#)
[Source Code and Binaries](#)

Crypto-js v4.0.0

Project Website
License
Download

[Crypto-js Website](#)
[Crypto-js License](#)
[Source Code and Binaries](#)

jsonwebtoken v8.5.1

Project Website
License
Download

[jsonwebtoken Website](#)
[MIT License](#)
[Source Code and Binaries](#)

React v16.13.1

Project Website
License
Download

[React Website](#)
[MIT License](#)
[Source Code and Binaries](#)

react-datepicker v3.3.0

Project Website
License
Download

[react-datepicker Website](#)
[MIT License](#)
[Source Code and Binaries](#)

react-hook-form v6.3.0

Project Website
License
Download

[react-hook-form Website](#)
[MIT License](#)
[Source Code and Binaries](#)

react-redux v7.2.1

Project Website
License
Download

[react-redux Website](#)
[MIT License](#)
[Source Code and Binaries](#)

react-router-dom v5.2.0

Project Website
License
Download

[react-router-dom Website](#)
[MIT License](#)
[Source Code and Binaries](#)

redux v4.0.5

Project Website
License
Download

[redux Website](#)
[MIT License](#)
[Source Code and Binaries](#)

redux-thunk v2.3.0

Project Website
License
Download

[redux-thunk Website](#)
[MIT License](#)
[Source Code and Binaries](#)

dgrijalva/jwt-go v3.2.0

Project Website
License
Download

[dgrijalva/jwt-go Website](#)
[MIT License](#)
[Source Code and Binaries](#)

gin-contrib/cors v1.3.1

Project Website
License
Download

[gin-contrib/cors Website](#)
[MIT License](#)
[Source Code and Binaries](#)

gin-contrib/gzip v0.0.2

Project Website
License
Download

[gin-contrib/gzip Website](#)
[MIT License](#)
[Source Code and Binaries](#)

gin-contrib/static v0.0.0-20191128031702-f81c604d8ac2

Project Website [gin-contrib/static Website](#)
License [MIT License](#)
Download [Source Code and Binaries](#)

gin-gonic/gin v1.6.3

Project Website [gin-gonic/gin Website](#)
License [MIT License](#)
Download [Source Code and Binaries](#)

go-delve/delve v1.6.0

Project Website [go-delve/delve Website](#)
License [MIT License](#)
Download [Source Code and Binaries](#)

go-openapi/errors v0.19.7

Project Website [go-openapi/errors Website](#)
License [Apache-2.0 License](#)
Download [Source Code and Binaries](#)

go-openapi/runtime v0.19.22

Project Website [go-openapi/runtime Website](#)
License [Apache-2.0 License](#)
Download [Source Code and Binaries](#)

go-openapi/strfmt v0.19.5

Project Website [go-openapi/strfmt Website](#)
License [Apache-2.0 License](#)
Download [Source Code and Binaries](#)

go-openapi/swag v0.19.9

Project Website [go-openapi/swag Website](#)
License [Apache-2.0 License](#)
Download [Source Code and Binaries](#)

go-openapi/validate v0.19.11

Project Website [go-openapi/validate Website](#)
License [Apache-2.0 License](#)
Download [Source Code and Binaries](#)

golang/protobuf v1.4.1

Project Website [golang/protobuf Website](#)
License [BSD 3-Clause](#)
Download [Source Code and Binaries](#)

google/gops v0.3.10

Project Website [google/gops Website](#)
License [BSD 3-Clause](#)
Download [Source Code and Binaries](#)

gorilla/websocket v1.4.2

Project Website [gorilla/websocket Website](#)
License [BSD 2-Clause](#)
Download [Source Code and Binaries](#)

josephspurrier/goversioninfo v1.4.0

Project Website [josephspurrier/goversioninfo Website](#)
License [MIT License](#)
Download [Source Code and Binaries](#)

kardianos/service v1.1.0

Project Website [kardianos/service Website](#)
License [Zlib License](#)

Download	Source Code and Binaries
labstack/echo v4.1.16	
Project Website	labstack/echo Website
License	MIT License
Download	Source Code and Binaries
phayes/freeport v0.0.0-20180830031419-95f893ade6f2	
Project Website	phayes/freeport Website
License	BSD 3-Clause
Download	Source Code and Binaries
satori/go.uuid v1.2.0	
Project Website	satori/go.uuid Website
License	MIT License
Download	Source Code and Binaries
smartystreets/goconvey v1.6.4	
Project Website	smartystreets/goconvey Website
License	MIT License
Download	Source Code and Binaries
spf13/pflag v.1.0.5	
Project Website	spf13/pflag Website
License	BSD 3-Clause
Download	Source Code and Binaries
stretchr/testify v1.6.1	
Project Website	stretchr/testify Website
License	MIT License
Download	Source Code and Binaries
Protobuf-go 1.25.0	
Project Website	protobuf Website
License	BSD 3-Clause
Download	Source Code and Binaries
gopkg.in/ini.v1 1.57.0	
Project Website	gopkg.in/ini.v1 Website
License	Apache 2
Download	Source Code and Binaries
FFmpeg 4.4	
Project Website	FFmpeg Website
License	FFmpeg License
Download	Source Code and Binaries
Assimp 5.0.1	
Project Website	Assimp Website
License	License
Download	Source Code and Binaries
QuaZip 1.1	
Project Website	QuaZip Website
License	Apache 2
Download	Source Code and Binaries

About licenses

MPEG-4 AVC:

THIS PRODUCT IS LICENSED UNDER THE AVC PATENT PORTFOLIO LICENSE FOR THE PERSONAL USE OF A CONSUMER OR

OTHER USES IN WHICH IT DOES NOT RECEIVE REMUNERATION TO (i) ENCODE VIDEO IN COMPLIANCE WITH THE AVC STANDARD ("AVC VIDEO") AND/OR (ii) DECODE AVC VIDEO THAT WAS ENCODED BY A CONSUMER ENGAGED IN A PERSONAL ACTIVITY AND/OR WAS OBTAINED FROM A VIDEO PROVIDER LICENSED TO PROVIDE AVC VIDEO. NO LICENSE IS GRANTED OR SHALL BE IMPLIED FOR ANY OTHER USE. ADDITIONAL INFORMATION MAY BE OBTAINED FROM MPEG LA, L.L.C. SEE [HTTP://WWW.MPEGLA.COM](http://www.mpegla.com)

Support

For further information on installing/exchanging the dynamic libraries please contact Circle Cardiovascular Imaging support.

Phone: +1 (403) 338-1870

E-mail: support@circlecvi.com

Website: circlecvi.com

Circle Cardiovascular Imaging Inc.

1100, 800 5th Avenue SW
Calgary, AB T2P 3T6
Canada