

macOS Quick Keys

cvi42 Platform

BASICS

Pan	drag
Zoom	+ + drag
Undo	+
Redo	+
Delete Selected	
Copy Selected	+
Copy All	+
Paste	+
Hide All	

Play Cine	
Refresh Study List	
Add to Case Review	+
Contours ON/OFF	+ +
Scroll Through Phases	+
Fullscreen	+
Preferences	+
View All Images in Multi-View	
Go to Next/Previous Series	/
Save Single Image as DICOM SC	
Save All Frames as DICOM SC	

WINDOW

Windowing	drag or + drag
Auto adjust	double click
Default	+
Presets	- ,

ANNOTATION TOOLS

Line Contour	
Freehand Contour	
Annotation Text	
Annotation Arrow	

Contours

DELETE

Delete All	+
Delete All (Slice)	+
Delete All (Phase)	+
Cut Selected	+
Delete Selected (Slice)	+
Delete Selected (Phase)	+

FORWARD

Forward All	+
Forward Selected	+
Select and Jump to Next	

MOVE

Drag	+ drag
Push	+ drag
Reposition All	+ drag

EDIT

Smooth	+
Round	+
Refine	hold
Click-Draw	+ click
Threshold Contour Definition	+ drag

cvi42 | Web Viewer

BASICS

Zoom



Pan



or shift +

Slice scrolling (coarse)



Slice scrolling (fine)



Window/Level



Adjust phase



or

Play cine




at bottom of viewport

or Press space

Rotate image



Maximize view

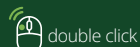

Next/previous series

On-screen arrow buttons ◀ ▶

or page up / page down

cvi42 | CORE CT

BASICS

Zoom



Pan



or shift +

Window/Level




Scrolling through MPR



Rotate volume



Rotate MPR

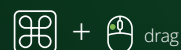


Re-center MPR views/ Triangulate MPR views to the 3D view



or click

Maximize viewport

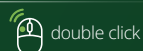

Context menu



CMPR CONTOURS

Scale contour



Edit lumen/wall simultaneously



Delete centerline control point



Quick Keys Valid for cvi42 v6.5