

Windows Quick Keys

cvi42 Platform

BASICS

Pan	 drag
Zoom	 +  + drag
Undo	ctrl + Z
Redo	ctrl + Y
Delete Selected	
Copy Selected	ctrl + C
Copy All	ctrl + A
Paste	ctrl + V
Hide All	shift

Play Cine	space
Refresh Study List	F5
Add to Case Review	ctrl + R
Contours ON/OFF	shift + ctrl + C
Scroll Through Phases	shift + 
Fullscreen	F11
Preferences	F12
View All Images in Multi-View	M
Go to Next/ Previous Series	page up / page down
Save Single Image as DICOM SC	F2
Save All Frames as DICOM SC	F3

WINDOW

Windowing	 drag or shift +  drag
Auto adjust	 double click
Default	ctrl + W
Presets	1 - 9, 0

ANNOTATION TOOLS

Line Contour	E
Freehand Contour	F
Annotation Text	T
Annotation Arrow	U

Contours

DELETE

Delete All	ctrl + 
Delete All (Slice)	ctrl + M
Delete All (Phase)	ctrl + P
Cut Selected	ctrl + X
Delete Selected (Slice)	ctrl + D
Delete Selected (Phase)	ctrl + O

FORWARD

Forward All	ctrl + E
Forward Selected	ctrl + F
Select and Jump to Next	tab

MOVE

Drag	D +  drag
Push	P +  drag
Reposition All	ctrl +  drag

EDIT

Smooth	ctrl + S
Round	ctrl + L
Refine	 hold
Click-Draw	C +  click
Threshold Contour Definition	Alt +  drag

cvi42 | Web Viewer

BASICS

Zoom



Pan



or



+



Slice scrolling (coarse)



Slice scrolling (fine)



Window/Level




Adjust phase



or



Play cine

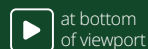

at bottom of viewport

or Press




Rotate image



+



Maximize view

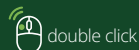

Next/previous series

On-screen arrow buttons ◀ ▶

or



/



cvi42 | CORE CT

BASICS

Zoom



Pan



or



+



Window/Level



Scrolling through MPR



Rotate volume



Rotate MPR



+



Re-center MPR views/ Triangulate MPR views to the 3D view




+



or



Maximize viewport

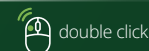

Context menu



CMPR CONTOURS

Scale contour



+



Edit lumen/wall simultaneously



+



Delete centerline control point



Quick Keys Valid for cvi42 v6.5