

Windows Quick Keys


cvi42 Platform

BASICS

Pan



Zoom

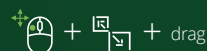

Undo



Redo



Delete Selected



Copy Selected



Copy All



Paste



Hide All



Play Cine

space

Refresh Study List



Add to Case Review



Contours ON/OFF



Scroll Through Phases



Fullscreen



Preferences



View All Images in Multi-View



Go to Next/ Previous Series



WINDOW

Windowing



Auto adjust



Default



Auto (Type 1)



Auto (Type 2)



Auto (Type 3)



Auto (Min Max)



Contours

DELETE


Delete All




Delete All (Slice)



Delete All (Phase)

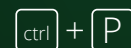


Cut Selected

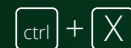

Delete Selected (Slice)



Delete Selected (Phase)

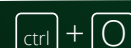

FORWARD

Forward All



Forward Selected



Select and Jump to Next




MOVE

Drag



Push

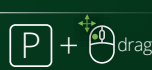


Reposition All




EDIT

Smooth

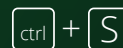

Round

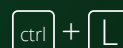

Refine



Click-Draw



Threshold Contour Definition



Windows and the Windows logo are registered trademarks of the Microsoft corporation in the United States and/or any other country.
Quick Keys Valid for cvi42 v6.x

cvi42 Web Viewer

BASICS

Zoom



drag

Pan



drag

or



+



drag

Slice scrolling (coarse)



drag

Slice scrolling (fine)



scroll

Window/Level



drag

Adjusting phase



drag

or



Play cine



at bottom
of viewport

or Press

space

Rotate image



+



Maximize view



double click

Next/previous series

On-screen arrow buttons



or

page
up

/

page
down

cvi42 | CORE CT

BASICS

Zoom



drag

Pan



drag

or



+



drag

Window/Level



drag

Scrolling through MPR



drag

Rotate Volume



drag

Rotate MPR



+



drag

Re-center MPR views



+



click

Maximize viewport



double click

Context Menu



click

Triangulate MPR Views to the 3D View



+



click in the 3D view

Windows and the Windows logo are registered trademarks of the Microsoft corporation in the United States and/or any other country.
Quick Keys Valid for cvi42 v6.x