F19AOS01

IT

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EN

SAFETY TIPS

Before installing and using the device, read the instructions for use and especially the safety warnings carefully and follow them. If in doubt, contact the company before using the device.

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EN

GENERAL INFORMATION

The device is safe to use, even for people with reduced visual, reading or cognitive abilities. There are no circumstances hampering ease of use of the product. The device must not be dismantled and does not have dismantable parts. The device's overall weight does not exceed 10 kg. The device does not contain carcinogenic, mutagenic or toxic substances. It is capable of withstanding corrosion and temperature variations.

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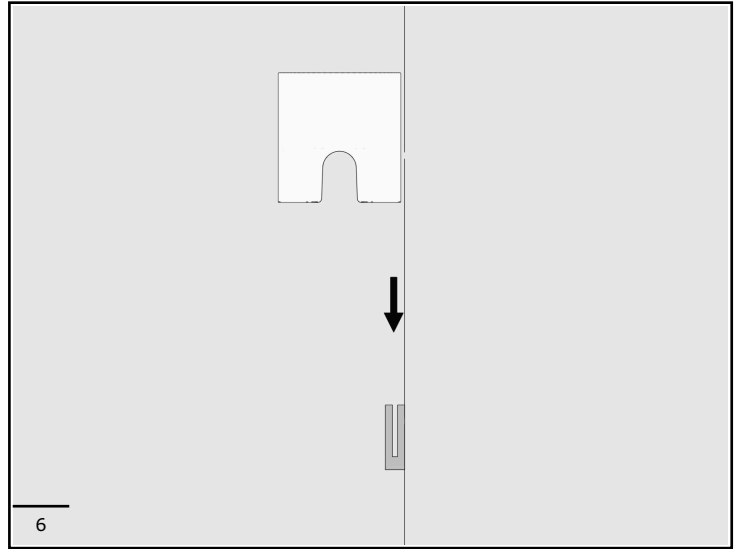
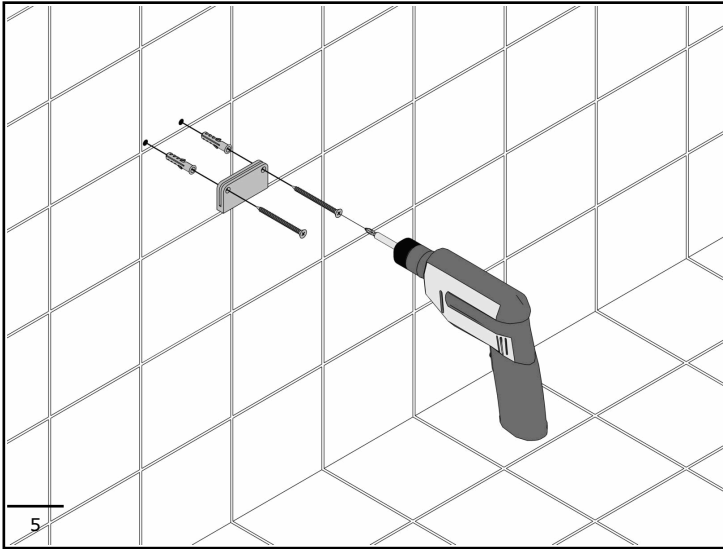
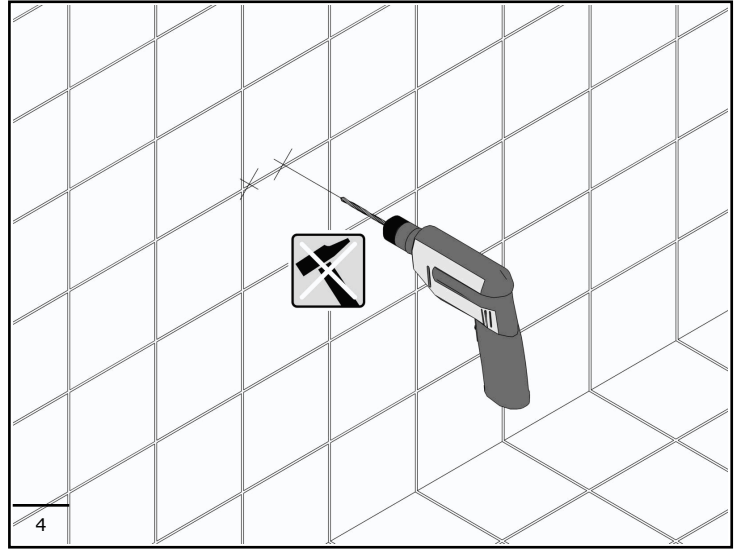
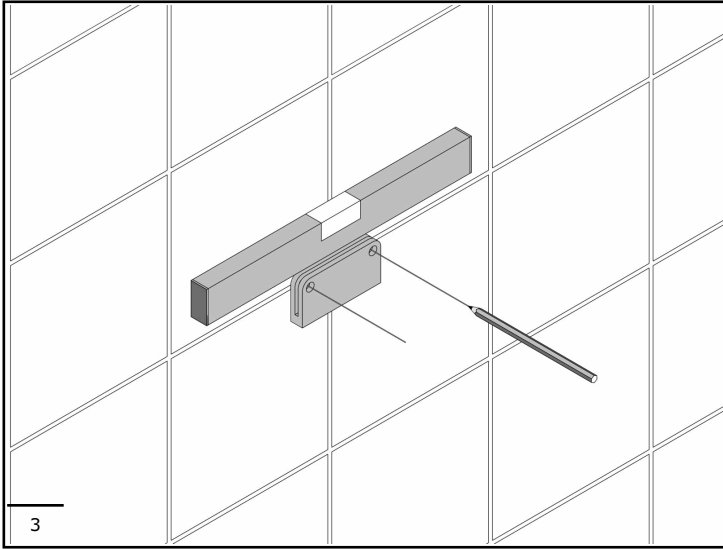
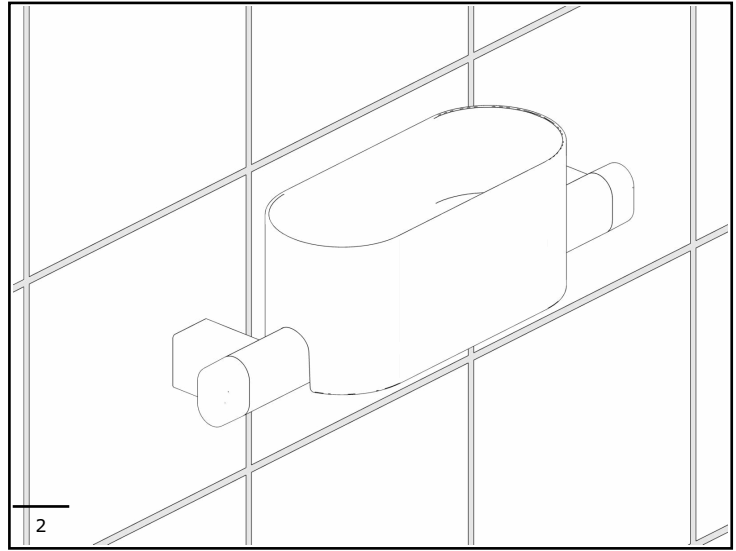
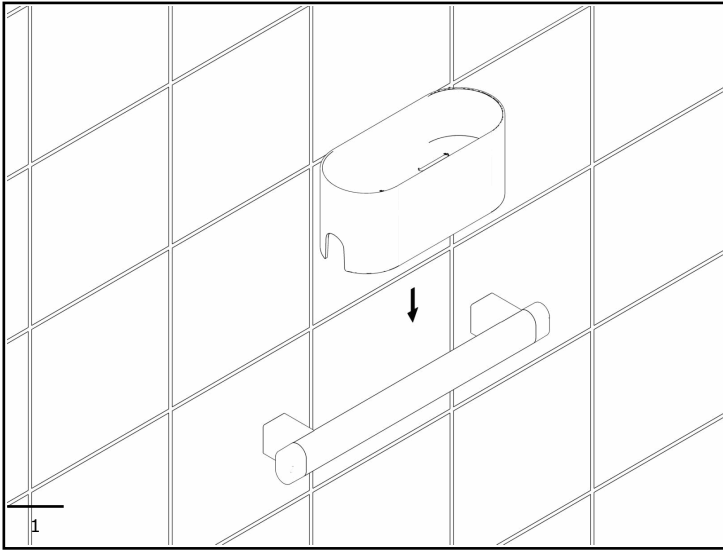
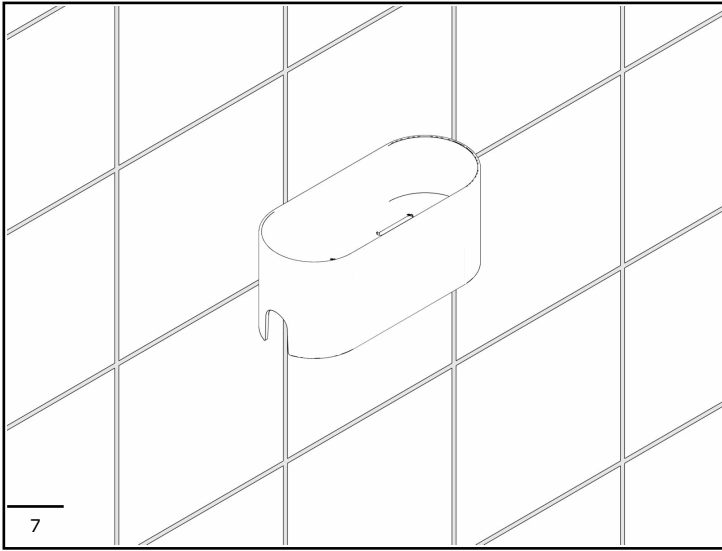
EN

The device is a product to be used exclusively as accessory. Using the device improperly, for example for gymnastics exercises, can cause injuries with serious consequences.
Do not subject the device to loads exceeding for which it was designed.
Exceeding this load can cause permanent damage to the device or harm to people.
To ensure correct use of the device, the latter must be left free of any objects and be easily reachable.

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EN

SAFETY PRECAUTIONS

The device must be installed by trained and qualified personnel (bricklayer or professional plumber) who must verify whether the fixing elements, if provided, are suitable for the relevant wall. To guarantee stability and safety, the device must be attached with wall plugs and screws suitable for the relevant type of wall.
Do not subject the device to loads exceeding those specified. If the device should become unstable, suspend its use until it is properly checked by the relevant technician.
The manufacturer declines all responsibility for installation of the device not conforming in this manual for use and maintenance.

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CLEANING

Clean the surface of the device using a damp cloth wetted with liquid neutral soap (e.g. dish soap). It is forbidden to use abrasive liquid detergents containing chlorine, strong acids or bases, concentrated solvents such as acetone and trichloroethylene or derivatives, which may cause the material too dull. It is forbidden to use hot air sources for drying. Disinfect the device's surface using a bleach-type product at least once a week.

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MAINTENANCE

Check the stability of the fixing at least every 3 months. Should the device become unstable, contact the installer. The device has no adjustments. It shall be fully replaced in case of any damages

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CONDITIONS OF USE

The device is not affected by the room's temperature, though it must NOT be installed in environments subject to temperatures below 4°C or above 40°C. Do not install in saunas.

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DISPOSAL

The user is responsible for the device's disposal, which must occur in accordance with the laws of the country in which the device

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is installed. The device is not manufactured using substances that are hazardous to human health and the environment, and all parts comprising it are recyclable and/or reusable. In order to safeguard the environment, the device also contains a certain amount of recycled materials.

Keep away from children packaging; risk of suffocation.

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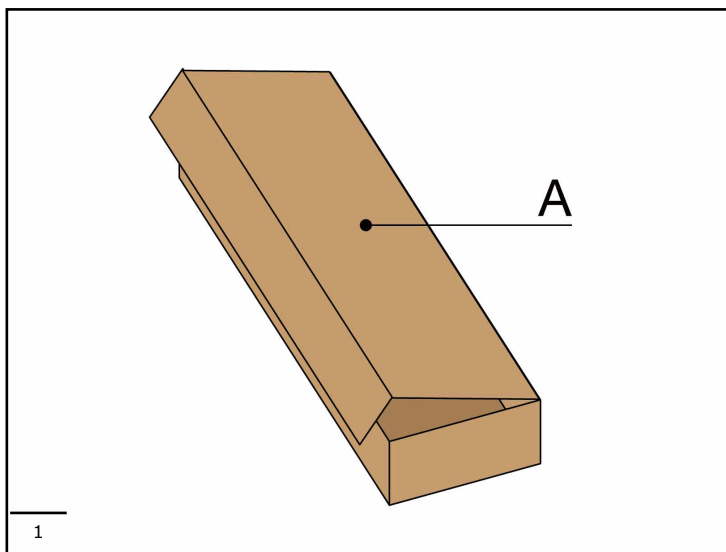
EN**FURTHER INFORMATION**

For further information, please visit our website or contact the our dealer zone

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EN**NOTE**

Olympia reserves the right to make changes to the products or to their specifications without prior approval or notification to the customer for:

- Changes required for safety purposes,
- Changes that do not adversely affect the product performance,
- Changes needed to meet the product specifications,
- Changes needed to comply with legal requirements or applicable regulations

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