

Keywing Key Turner Instructions



The **Keywing key turner** makes it twice as easy to turn a key. The extra leverage halves the strength required to twist a key. It instantly converts keys into an easy grip thumb turn and may help reduce hand pain.

Key Features

- Safety & Pain Relief: Halves the strength required to grip & turn keys.
- Ease of Use: Stops you dropping, fumbling and losing keys.
- Multi Purpose: Easily find & identify keys anywhere.

How to Use

- 1 Slide your key securely into the Keywing holder.
- 2 Ensure the key is fully inserted and clipped into place.
- 3 Hold the Keywing grip comfortably between your fingers and thumb.
- 4 Insert the key into the lock.
- 5 Turn the key using the added leverage from the Keywing grip.
- 6 Remove the key and store safely on your keyring.

The perfect gift for those with arthritis, Parkinson's, MS or weak hands.

Keywing – Easy Grip Key Turner