


Shower Chair and Bathroom Commode Portable Lightweight Fold Up Aluminum 150kg

Aqua/Brushed - S03C3

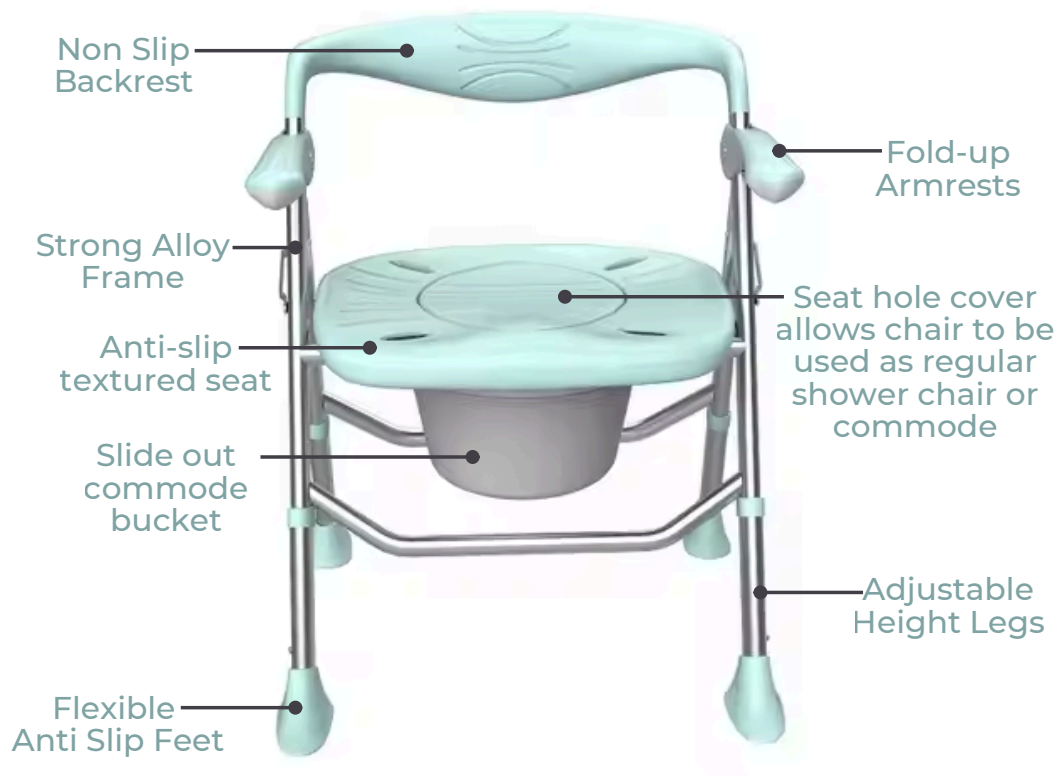

Note: This comes assembled ready to use.

AVAIL QUALITY DESIGNS

- Folds up for storage or transport
- 150kg Capacity
- Fold up arm rests
- Adjustable height legs
- Non slip feet
- Removable commode bucket
- Seat hole cover allows chair to be used as regular shower chair or commode



WWW.AVAIL.DESIGN



INFO@AVAIL.DESIGN



1300 945 720

Shower Chair and Bathroom Commode Portable Lightweight Fold Up Aluminum 150kg

Aqua/Brushed - S03C3



Product weight 4kg

Avail Fold Up Shower Chair And Commode

Aqua/Brushed - S03C3



Product Information:

This product comes with a 10 year manufacturers warranty. For any service or repair enquiries contact info@avail.design 1300 945 720 or our online form: avail.design/pages/contact. The product ID number is located on the inner front left leg.

Intended Use Instructions:

The intended operator for unfolding the chair can be either the occupant or assistant. The chair can be used with or without the commode bucket.

Arm Support Adjustment:

The armrest can be lifted up to vertical position to allow for easier access when transferring on and off the chair, also for cleaning and transport, they do not lock into place, the chair arm supports can only support the user when they are positioned horizontally.

Leg Height Adjustment:

When the legs are fully extended they increase the footprint of the chair, please check the dimensions to ensure they are suitable for your space. The leg height can be adjusted by pushing and holding in the spring loaded buttons on the legs then simultaneously extending the leg in or out. Adjust 1 leg at a time.

Chair Folding Instructions:

When folding the chair grip the seat at the centre front edge and the backrest at the centre and collapse them towards each other. There is no locking mechanism when folded or unfolded. To transport chair the preferred locations to carry the products are from the handle in the backrest, or if on its side then by the legs.

Maintenance:

Regular cleaning is required to reduce the chance of foreign material building up which can encourage mould and bacteria to grow. The chair has been approved to international standards to ensure bacteria does not grow on the materials however regularly cleaning is still required. The commode bucket slides forward to remove it and for cleaning.

This product does not require servicing and is intended to last at least 10 years before requiring replacement. If at any point you feel the product is not performing correctly please contact Avail and supply photos if possible.

If you notice any damage to the product please contact discontinue use and contact Avail.

Cleaning Method:

Warm soapy water or bathroom cleaning spray is recommended. Use a scrubbing brush around the edges, hinges, feet to ensure material doesn't accumulate. Automatic washing machines may be suitable. Bleach may be suitable however please be aware this may discolour the seat and arms.

Warnings:

Chair legs are a potential trip hazard. Ensure you have adequate circulation space around the product to reduce the chance of tripping.

The arm supports have potential pinch points when lowering them to the horizontal position, ensure your hands and body parts are away from the hinge area when lowering the arms, preferably hold on at the end furthest from the hinge.

When folding the chairs there is a potential pinch point at the hinge, ensure your hands and body parts are away from the hinge when folding the chair.

Ensure all four legs are even height and are on the floor before sitting. Ensure hands are holding onto both handles when sitting/standing to avoid the chair tipping.

