


Furine Vet


Application Guide


Preparation:

1 Remove hair from the perianal skin to allow attachment of adhesive base.




2 Cleanse and dry the perianal skin. Ensure the skin is completely dry and intact. Prepare the skin using **Securin Adhesive Remover Wipe** to ensure firm adhesion. **Do not** use any other skin barrier film products.

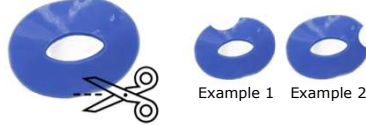


3  Example 1 Example 2

If necessary, adapt the adhesive base to fit the patients' anatomy.


Application:

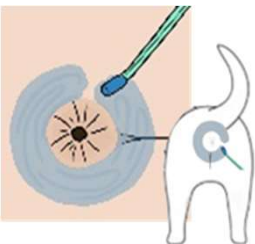
4  Apply Securin Skin Barrier to the exposed skin around the anus using the swab stick.

- Apply in a width slightly larger than the size of the adhesive base.
- **NOTE:** The entire adhesive base must adhere to skin treated with Securin Skin Barrier.


5  Allow the Securin Skin Barrier to dry for 1-2 minutes (**until dry**). Hold the tail up, if necessary.


- **Important:** Securin Skin Barrier must be **completely dry** before applying Furine Vet.
- Application to wet or partially dry Securin Skin Barrier will result in reduced adhesion and leakage.

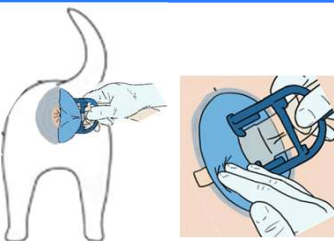6  Using the applicator, position Furine Vet around the anus on the perianal skin where Securin Skin Barrier has been applied. Press the entire adhesive base firmly onto the skin

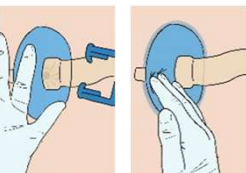7  Carefully remove the applicator, while holding Furine Vet in place with the other hand.

- Press the adhesive base firmly **again** to ensure full contact.

8  Remove excess Securin Skin Barrier from surrounding skin using the second Securin Adhesive Remover Wipe provided.

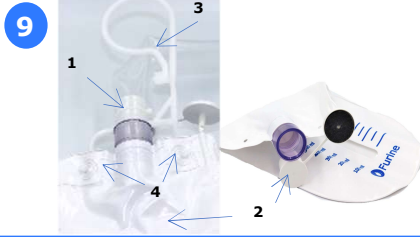

Attach Furine Collect (2) by pressing it on to the outlet of the tube (1). Insert the safety hooks (4) into the holes in Furine Collect. The hanger (3) can be used to position the bag on the side of the cage, if appropriate.


Replace Furine Collect daily, or more frequently when needed.

Maintenance:

Regular maintenance optimizes wear-time and performance.

Maintenance comprises the following steps should be done every 2 hours, and includes:

- Ensure the tube is not kinked or stretched, and that free passage is maintained
- Press firmly on the entire surface of the adhesive base to reinforce adhesion
- Inspect for leakage. If leakage is detected, repair using Securin Skin Barrier

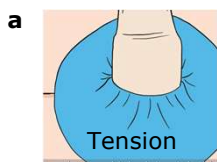

Removal:



- Remove Furine Vet using Securin Adhesive Remover Wipe and/or Spray as recommended by Furine ApS.
- Gently lift an edge of the adhesive base while wiping with 2-3 wipes and/or spraying adhesive remover.
- Carefully remove all residue of Securin Skin Barrier from the skin

Do not

- use wipes that have dried out
- use wipes that are not siloxane-based.

Disposal:

Dispose of used Furine Vet, Furine Collect, and accessories as biohazardous clinical waste, in accordance with local regulations and procedures.



Video Tutorial

You are welcome to contact us at veterinary@furine.com

Read more about Furine ApS' products at www.furinevet.com

Furine Vet is for veterinary use only. Not for human use.