

Product Description & Intended Use

Furine[®] Vet Collect is an accessory to Furine[®] Vet. The intended use of Furine Vet Collect is to collect liquid fecal matter from patients with fecal incontinence using Furine Vet.

Furine Vet Collect can contain 1000 ml of liquid.

Furine Vet Collect includes a charcoal filter to allow air to escape and to reduce odour and it also includes a gel-blocker in the air outlet tube to prevent leakage of fecal matter from Furine Vet Collect. Furine Vet Collect should be replaced daily and then discarded.

The device is intended to be used by trained veterinary professionals in a veterinary medical care setting.

Indications

Furine Vet Collect is indicated to collect fecal liquid from Furine Vet.

Warnings and Precautions

Single-use device. Do not reuse or re-process. Reuse may result in infection or cross-contamination. Do not sterilize. If there is fecal odour emanating from Furine Vet Collect, it should be replaced. If the odour continues after replacement of Furine Vet Collect, Furine Vet should be replaced. A Furine Vet Collect containing liquid should be kept in an upright position to avoid fecal matter in the air outlet tube. If liquid matter enters the air outlet tube, the gel-blocker in the air outlet tube will block the outlet, preventing escape of air, and Furine Vet Collect must then be replaced.

Furine ApS is not responsible for damage or injury resulting from use outside the instructions for recommendations provided.

Additional information

MRI safety has not been evaluated. Not intended for use in human MRI environments.

To learn more or for technical assistance please contact us through www.furine.com.

Instructions for Use

1. Attach Furine Vet Collect to the connector of Furine Vet by pushing the opening of Furine Vet Collect onto the opening of the connector.
2. Place the two safety hooks of the connector into the two holes in Furine Vet Collect, so that Furine Vet Collect does not unintentionally fall off.
3. Allow Furine Vet Collect to remain attached for a maximum of 24 hours before being replaced, or replace Furine Vet Collect, when it contains 1000ml of fluid.
4. Furine Vet Collect is replaced by removing Furine Vet Collect from the connector of Furine, and then placing the lid of Furine Vet Collect firmly into the opening of Furine Vet Collect.
5. Furine Vet Collect can then be disposed of. Dispose of Furine Vet Collect according to local regulations.



Consult Instructions for Use



Use-by date
See primary packaging



Legal Manufacturer:
Furine ApS
Industrivaenget 1
3320 Skaevinge
Denmark



30°C
5°C Storage conditions



Batch code
See primary packaging



Reference code:
See primary packaging



Single Use



Latex free

Version: IFUV205-01US 2026-01